
MOUNTING- INSTRUCTION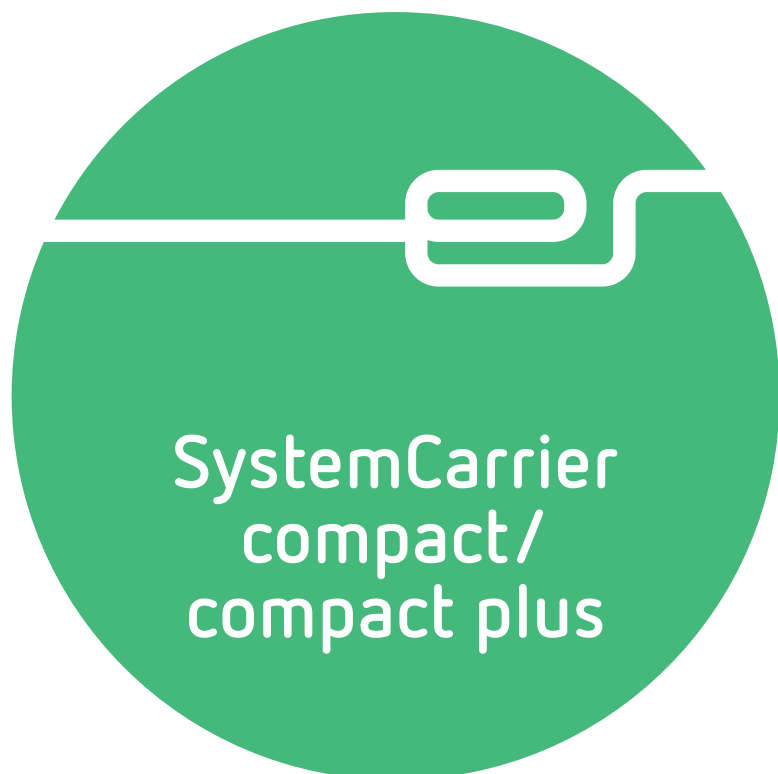

EN ISO 9001 EN ISO 13485

SystemCarrier compact Art. No. 20180-100, SystemCarrier compact plus Art. No. 20180-101

All rights to this Mounting Instruction, in particular rights of duplication, dissemination and translation, are reserved. No part of this Mounting Instruction may be reproduced in any form (by photocopying, microfilming or other methods) or processed, duplicated or disseminated by the use of electronic systems without the written consent of Erbe Elektromedizin GmbH.

The information contained in this Mounting Instruction can be changed or expanded without prior notice and without obligation on the part of Erbe Elektromedizin GmbH.

The illustrations in this Mounting Instruction may differ in some details from your product.

Printed by Erbe Elektromedizin


Printed in Germany

Copyright © Erbe Elektromedizin GmbH, Tübingen 2023

What would you like to do?

 Click on the accessory / unit to be mounted.

 Caution, read safety instructions before installation

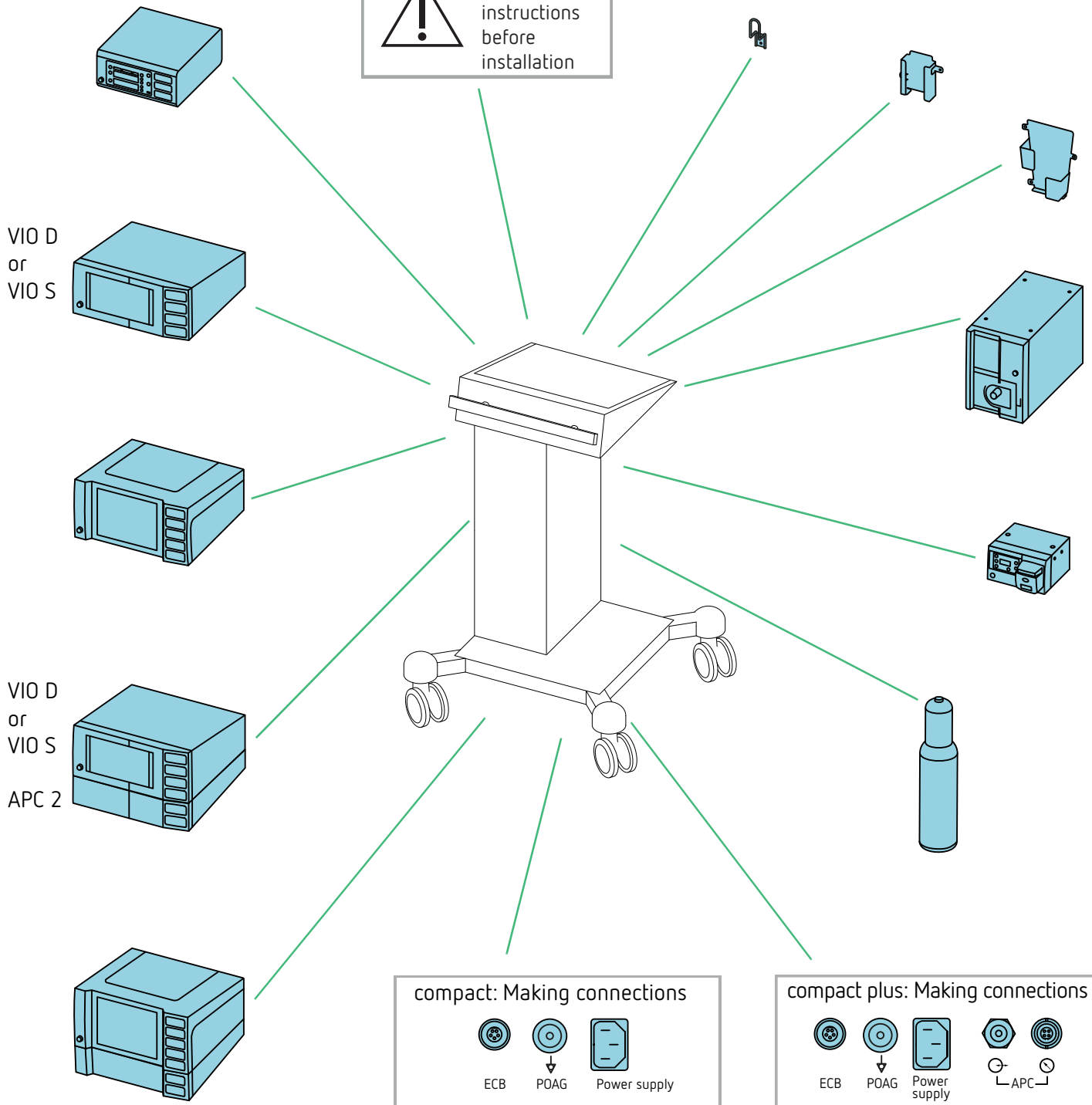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

Normal use

Permitted installation points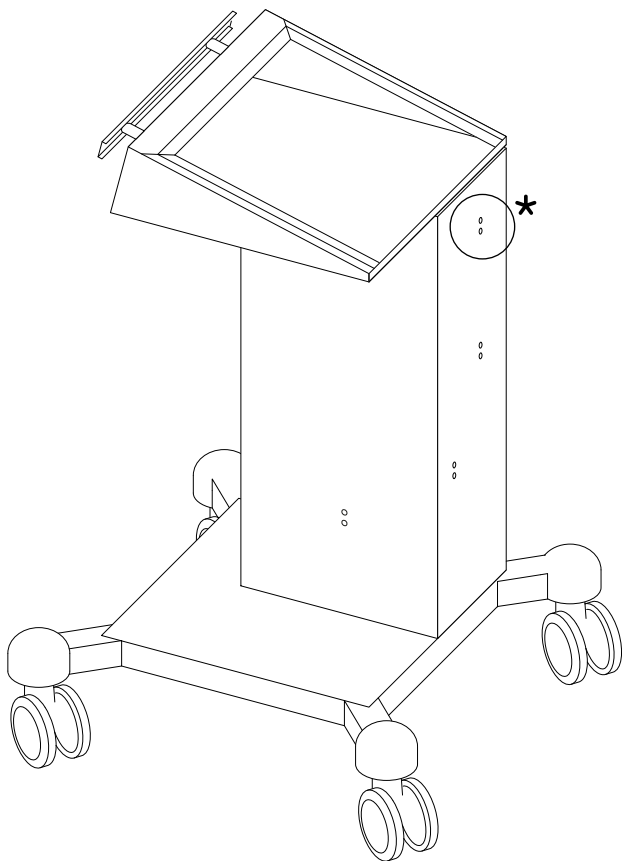

Maximum loading:
For compact, 4 items
for compact plus, 3 items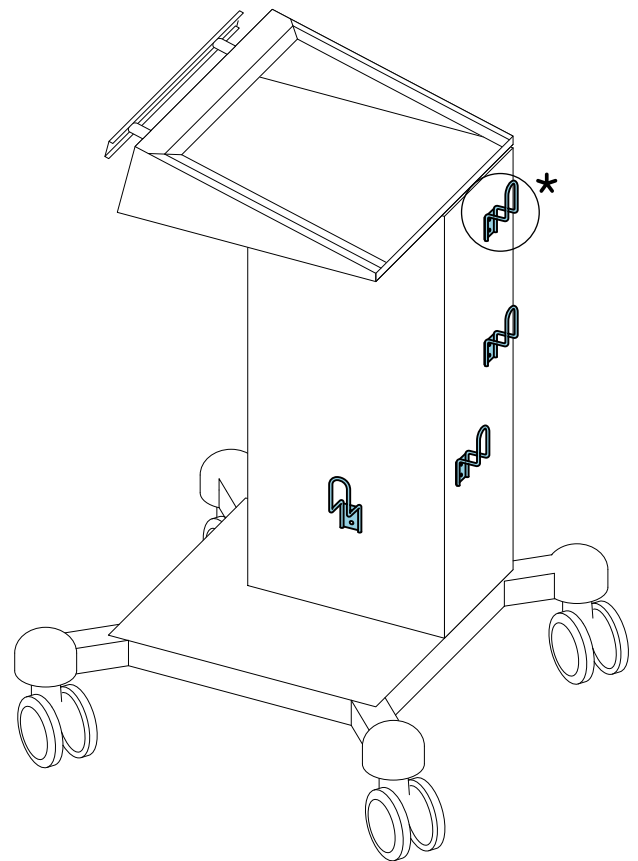


★ Omitted for compact plus

Scope of delivery

IMPORTANT! Only use original Erbe accessories.

Erbe REF 20180-460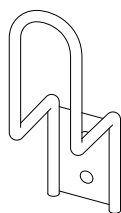

M4 x 8





Tools required

Torque screwdriver (1 Nm) hex socket 2.5

Cutter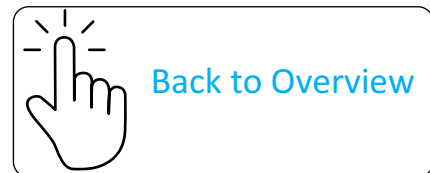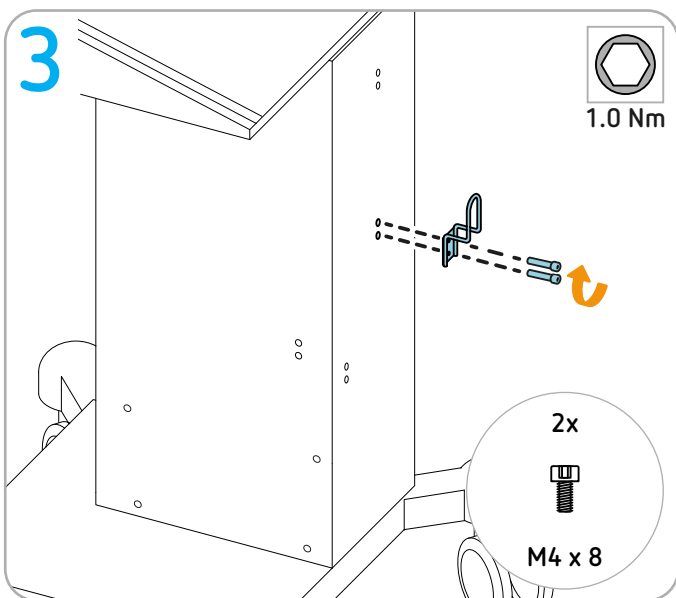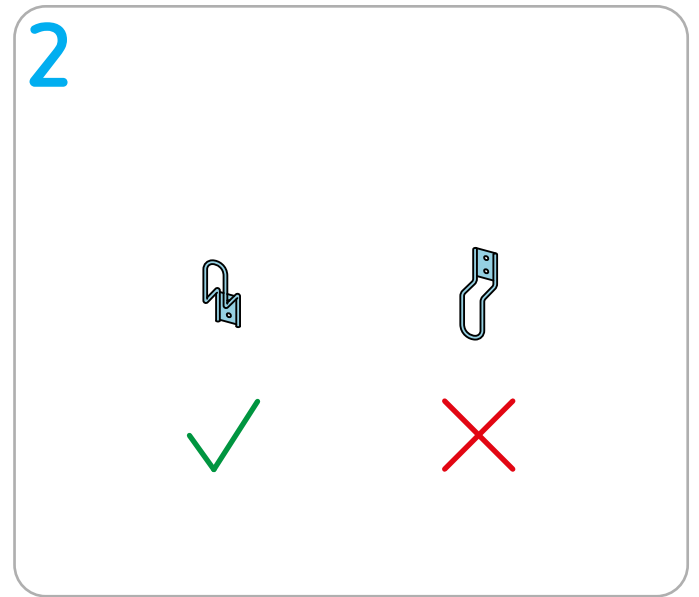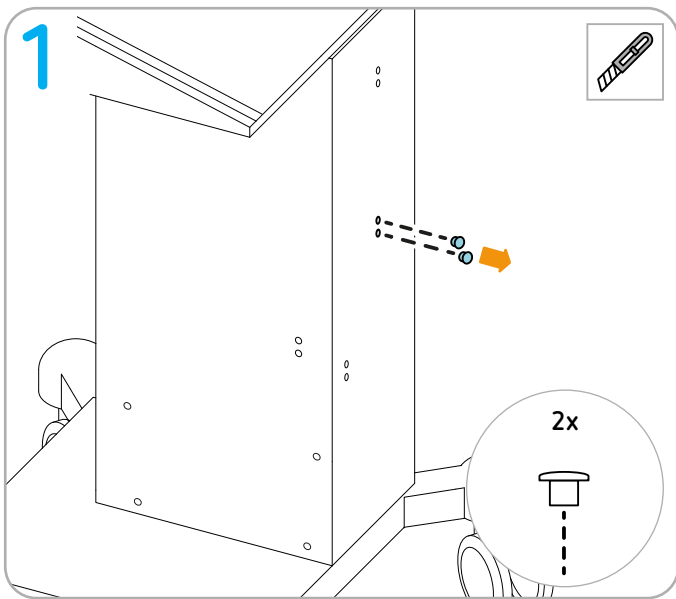


Installing footswitch support for large one-pedal footswitch

Normal use


This footswitch support is compatible with the following Erbe one-pedal footswitches::

- VIO S, VIO D and VIO 3 one-pedal footswitch
- IES 3 one-pedal footswitch

Permitted installation points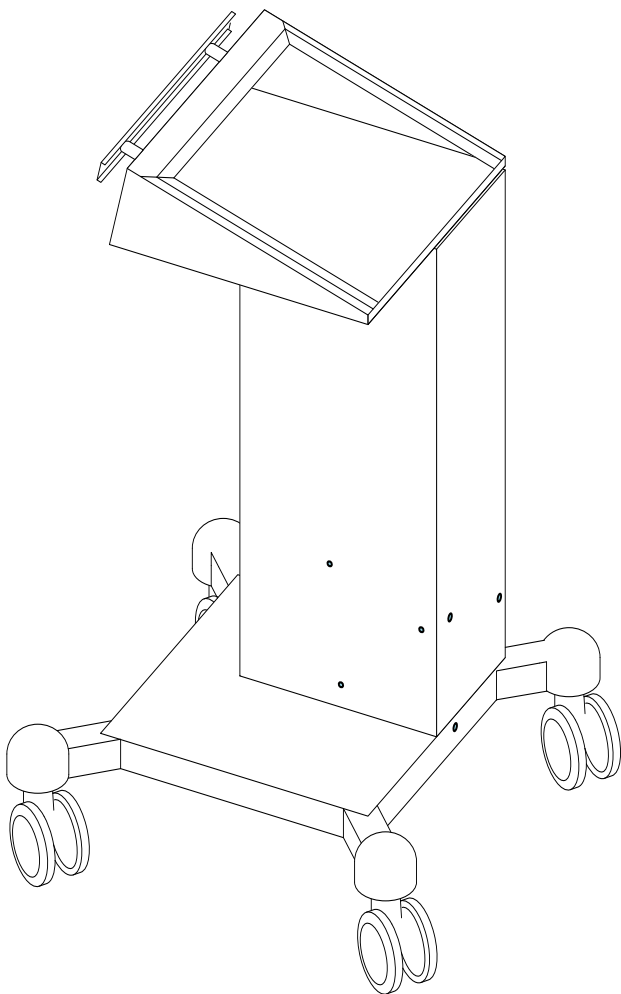


Maximum loading: 2 items

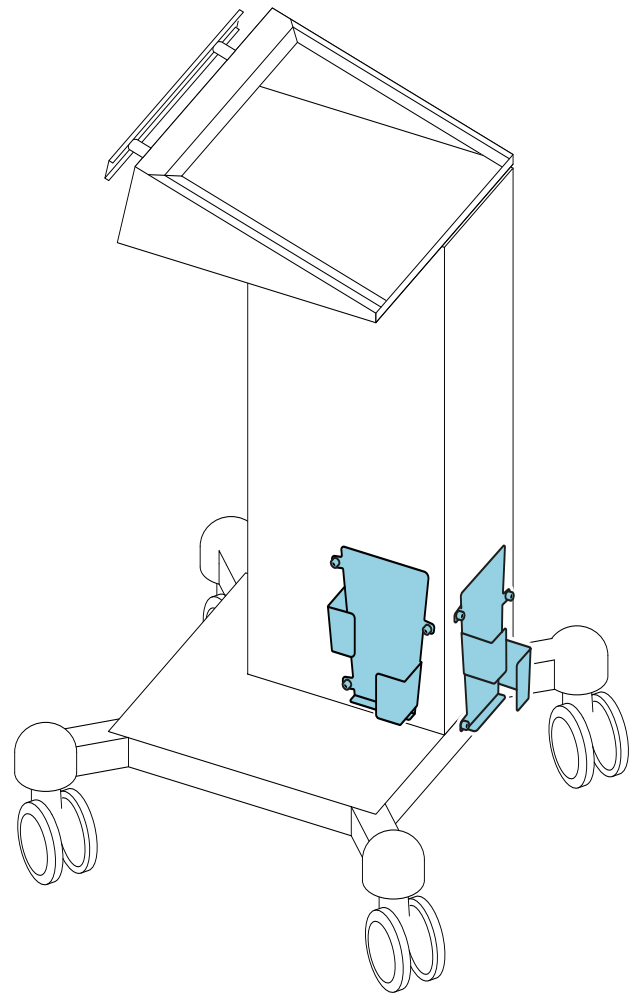Scope of delivery

IMPORTANT! Only use original Erbe accessories.





Erbe REF 20180-463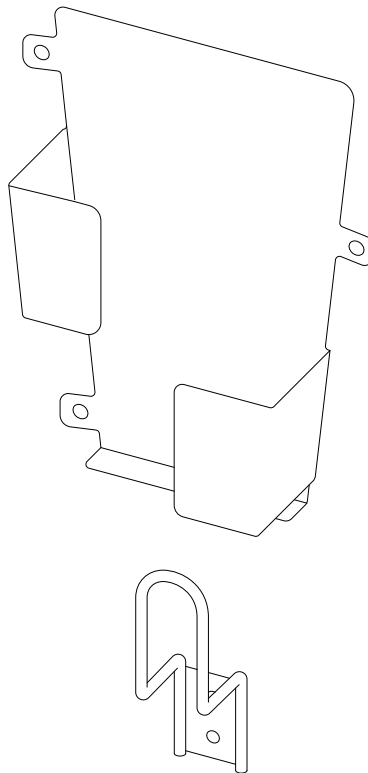

Tools required

 Torque screwdriver (1 Nm) hex socket 2.5

 Cutter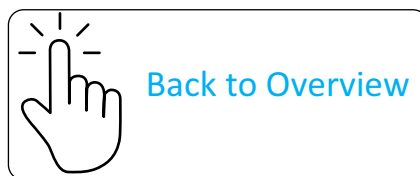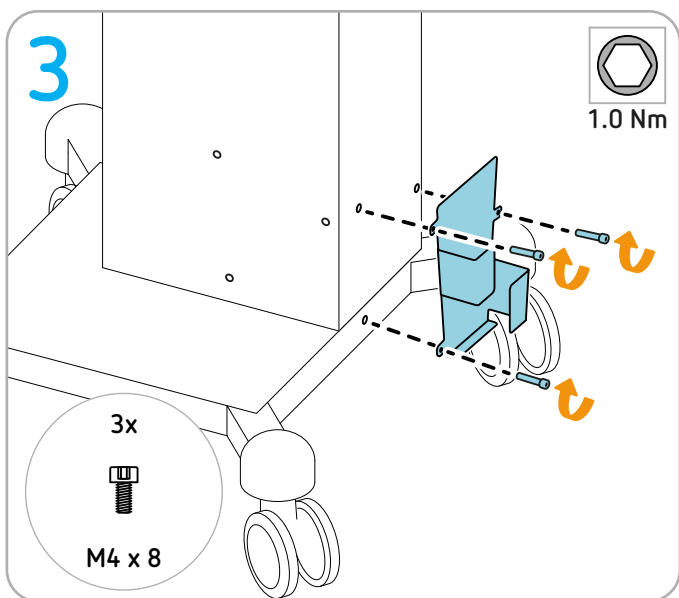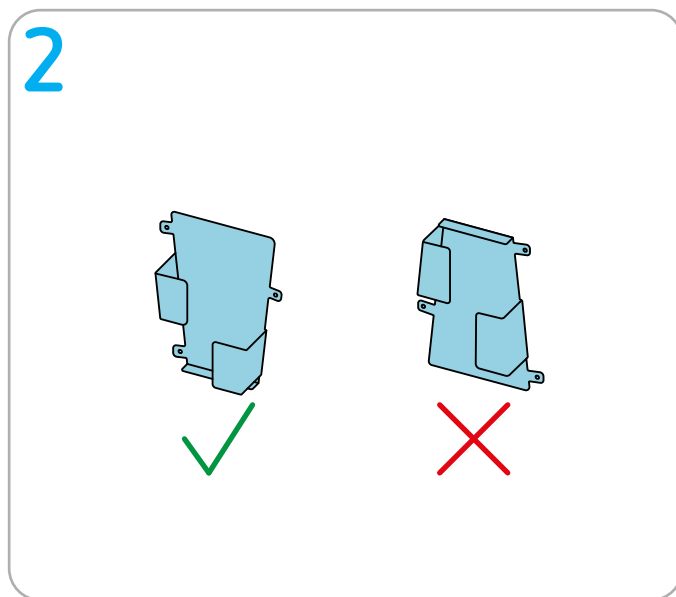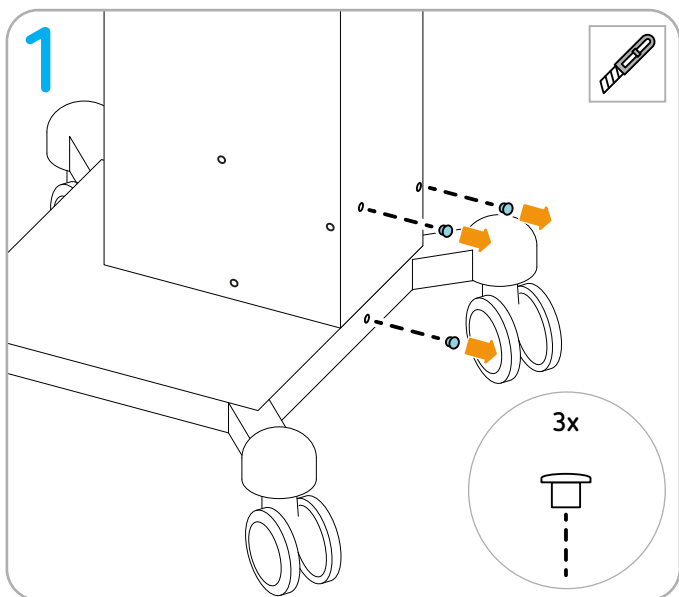


Installing footswitch support for small one-pedal footswitch

Normal use


This footswitch support is compatible with the following Erbe one-pedal footswitches::

- VIO C one-pedal footswitch
- EIP 2 one-pedal footswitch

Permitted installation points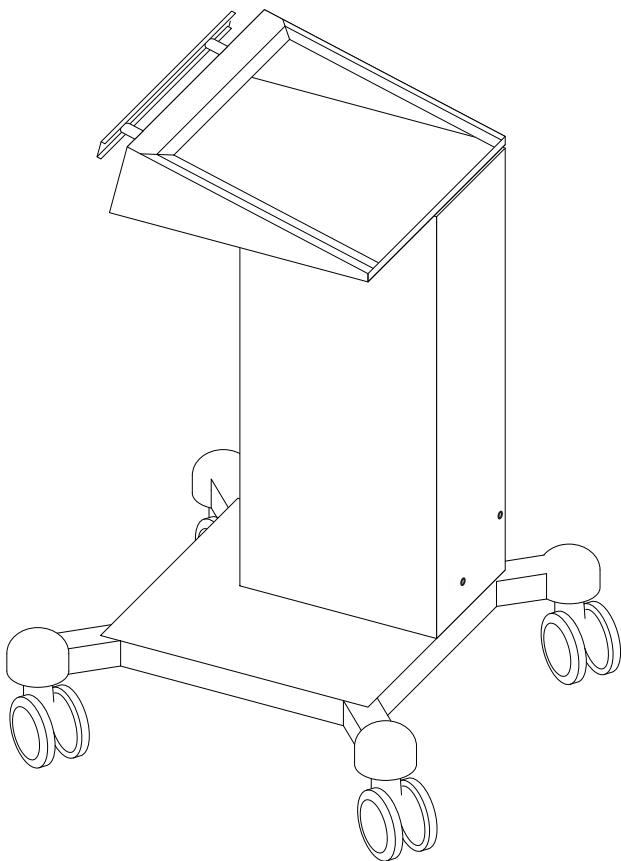



Maximum loading: 1 item

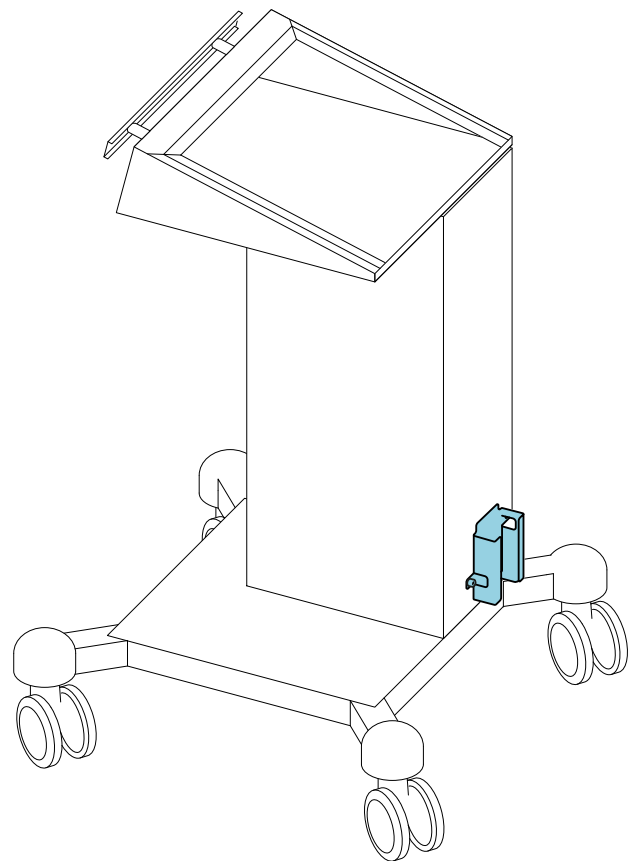Scope of delivery

IMPORTANT! Only use original Erbe accessories.





Erbe REF 20180-464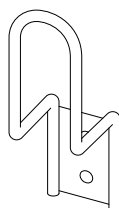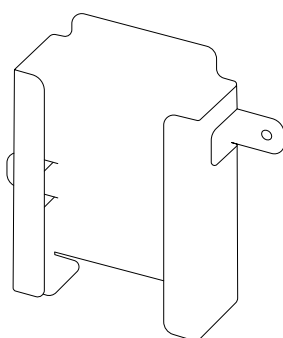

Tools required


 Torque screwdriver (1 Nm) hex socket 2.5

 Cutter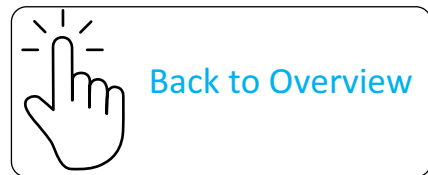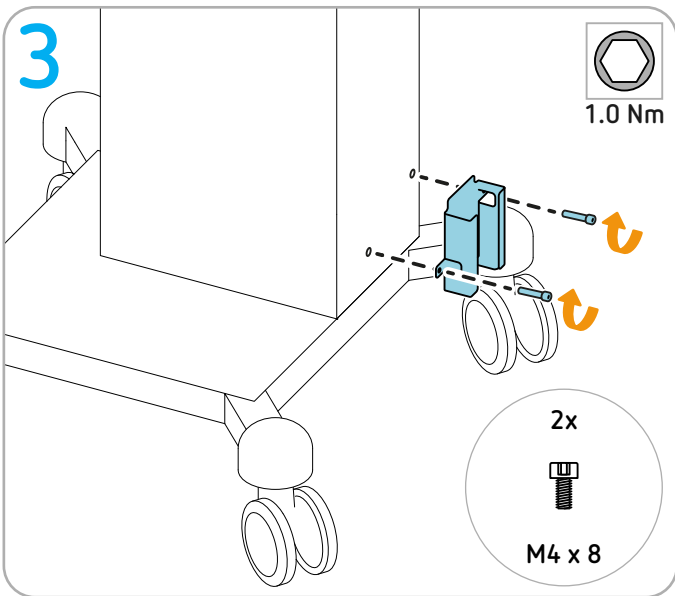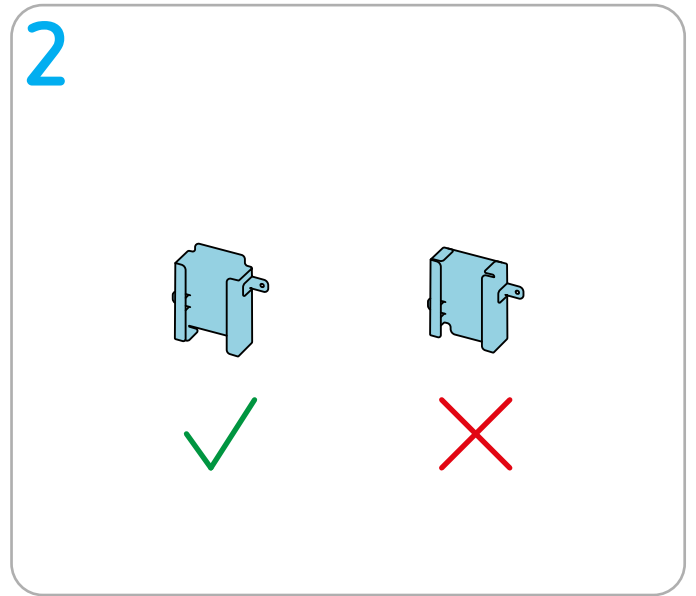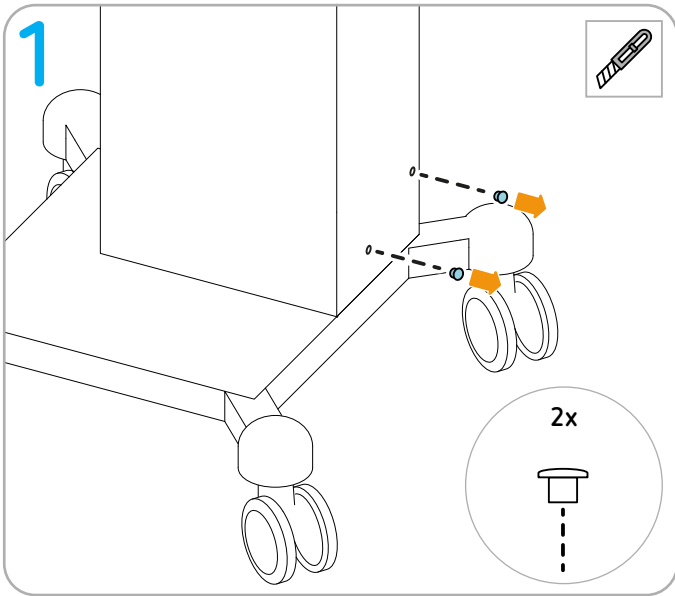


Installing IES 3

Permitted positioning

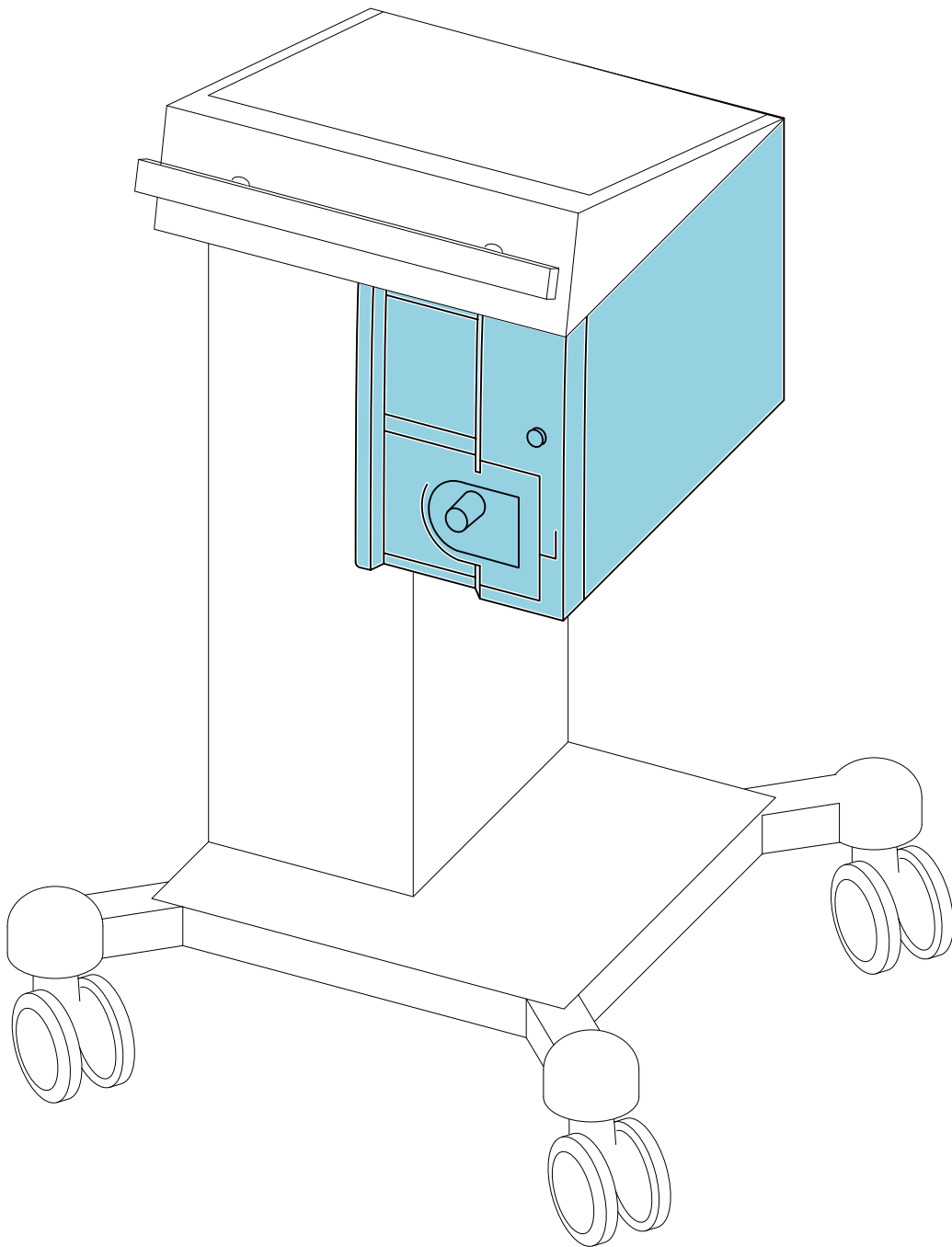Erbe fastening set

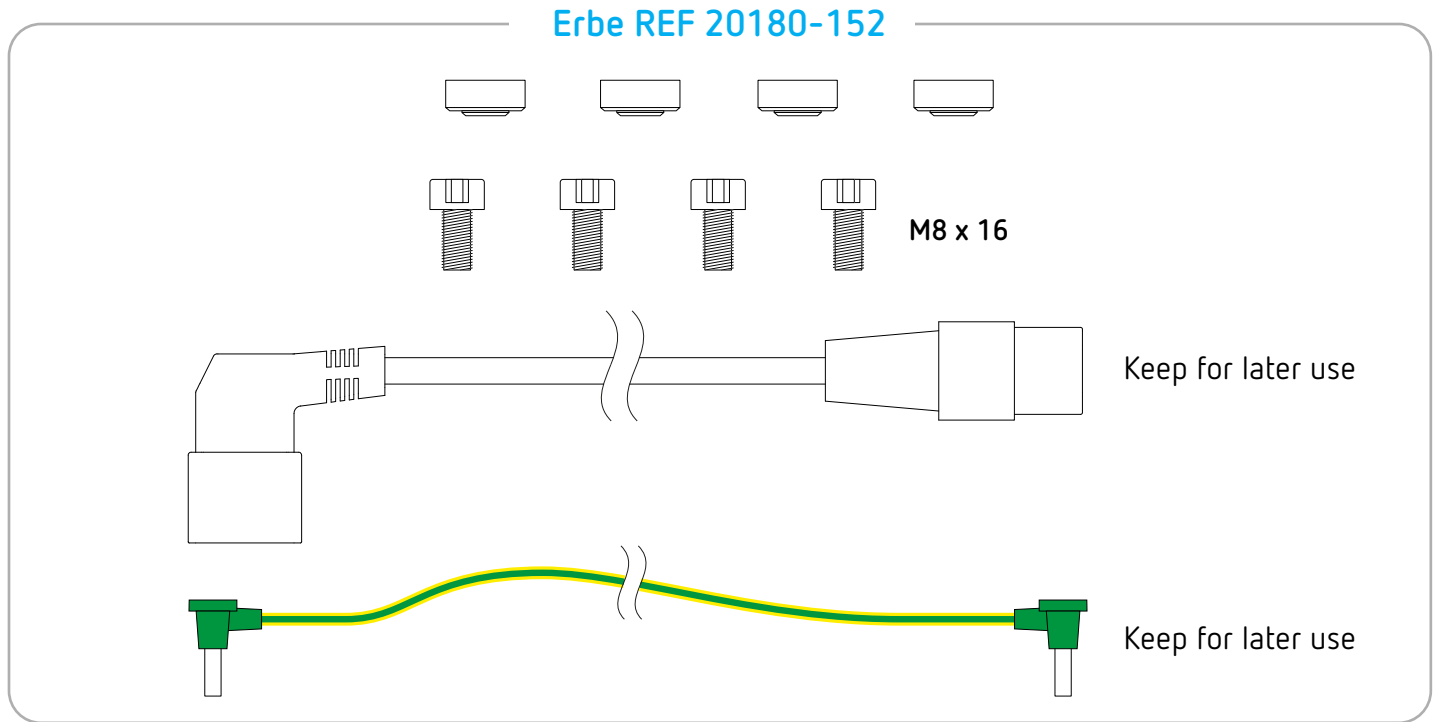IMPORTANT! Only use original Erbe accessories.







Tools required

-  Torque screwdriver (1,5 Nm) hex socket 6
-  Torque screwdriver (1,5 Nm) hex socket 2,5
-  Cutter


Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.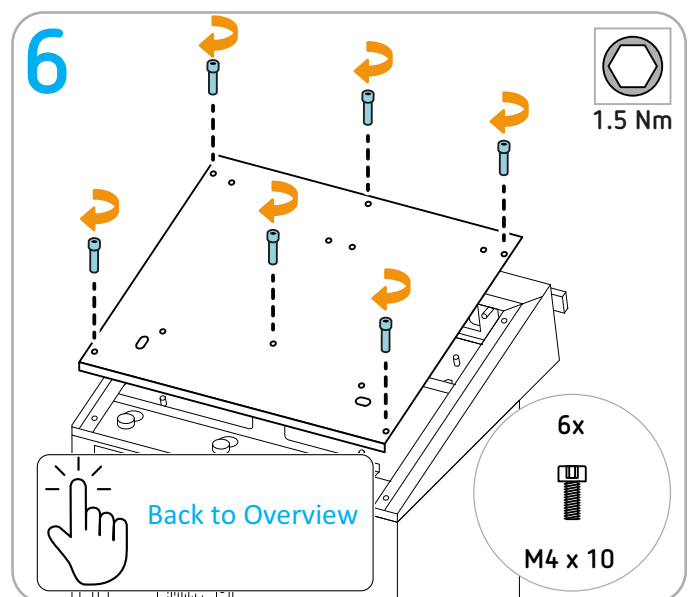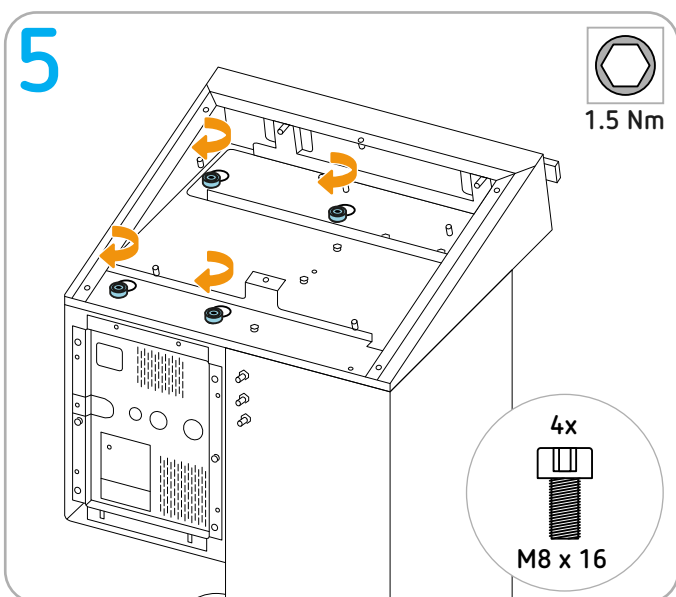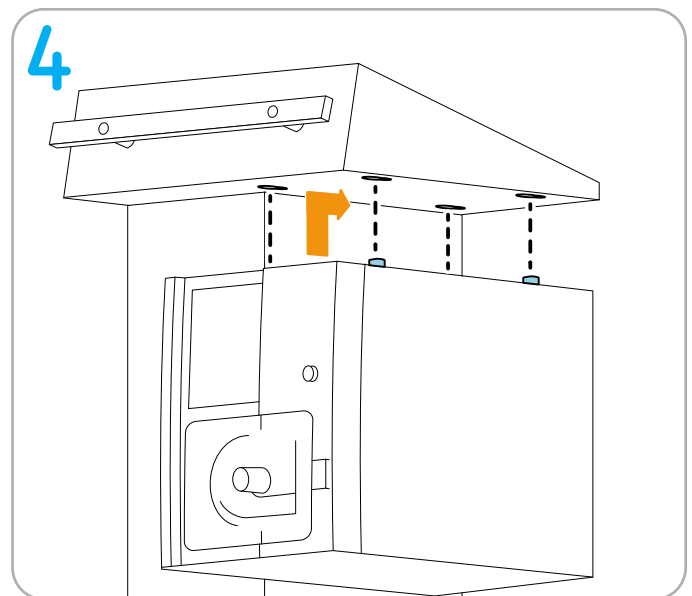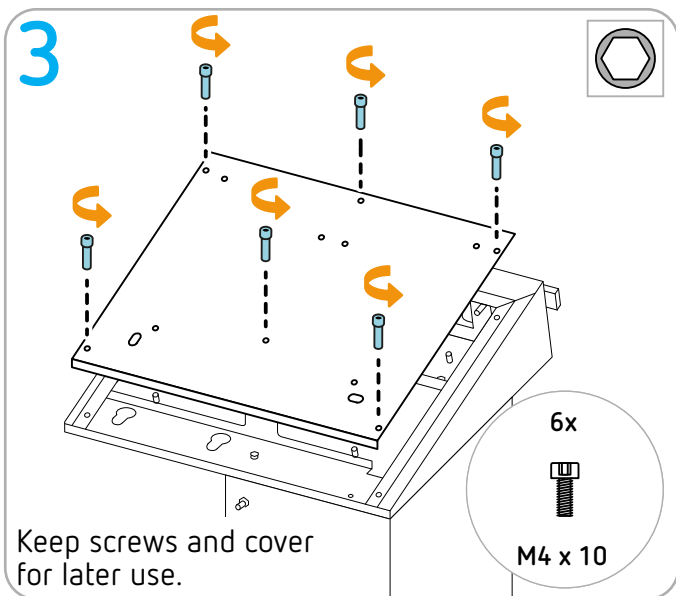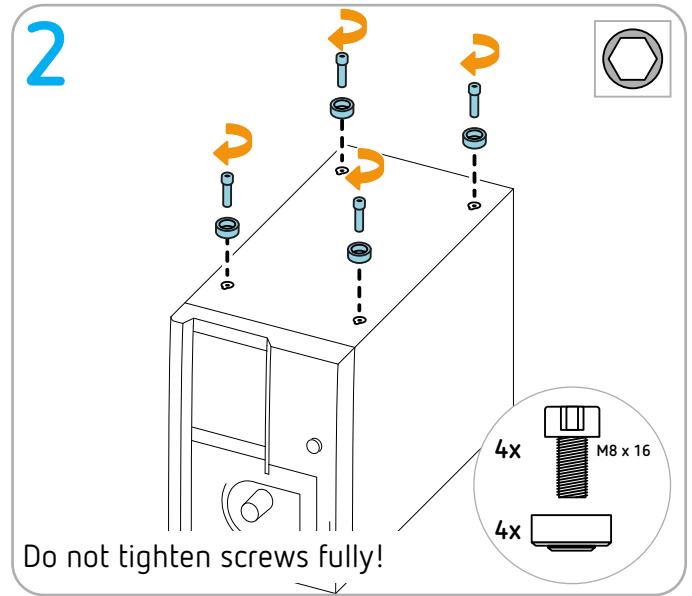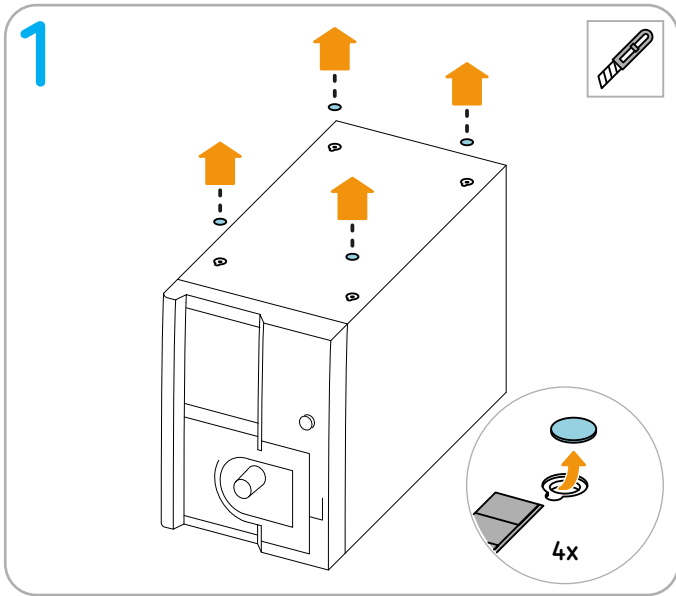


Installing EIP 2

Permitted positioning

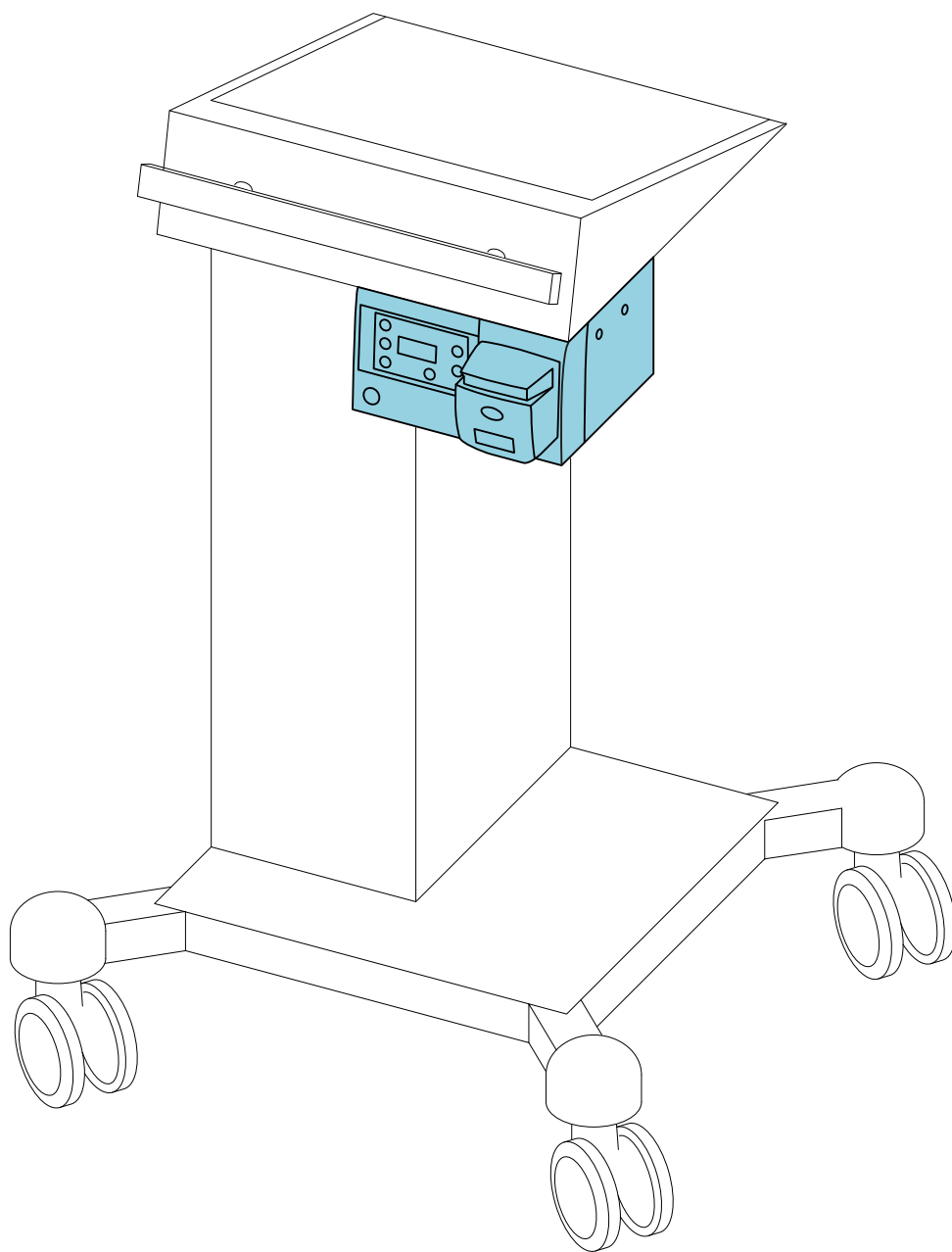Erbe fastening set

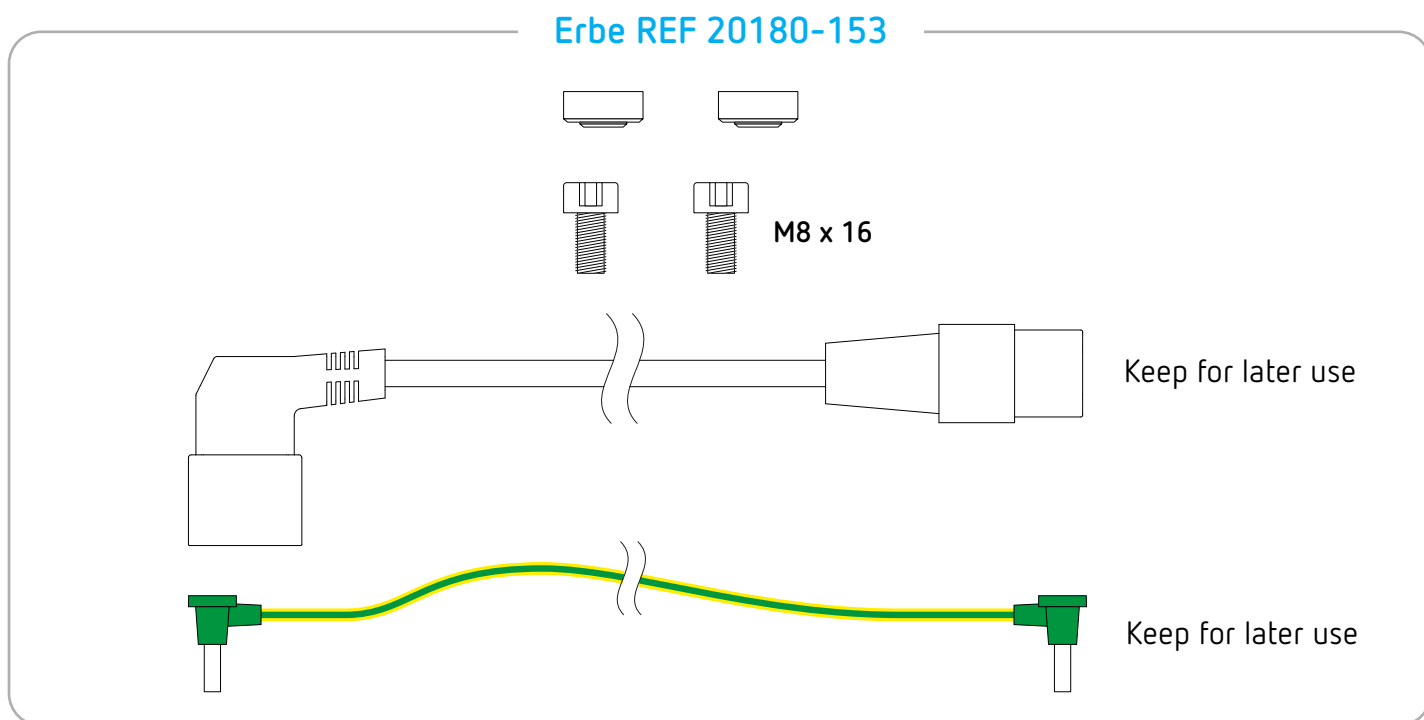IMPORTANT! Only use original Erbe accessories.







Tools required

-  Torque screwdriver (1,5 Nm) hex socket 6
-  Torque screwdriver (1,5 Nm) hex socket 2,5
-  Cutter


Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.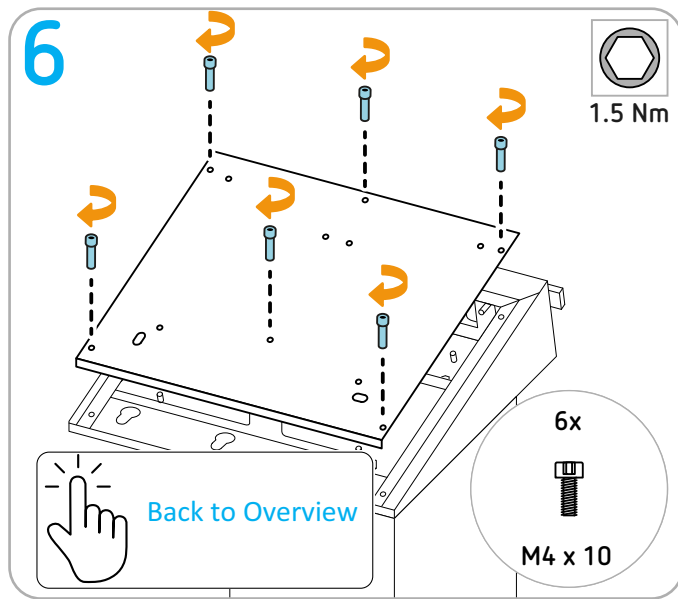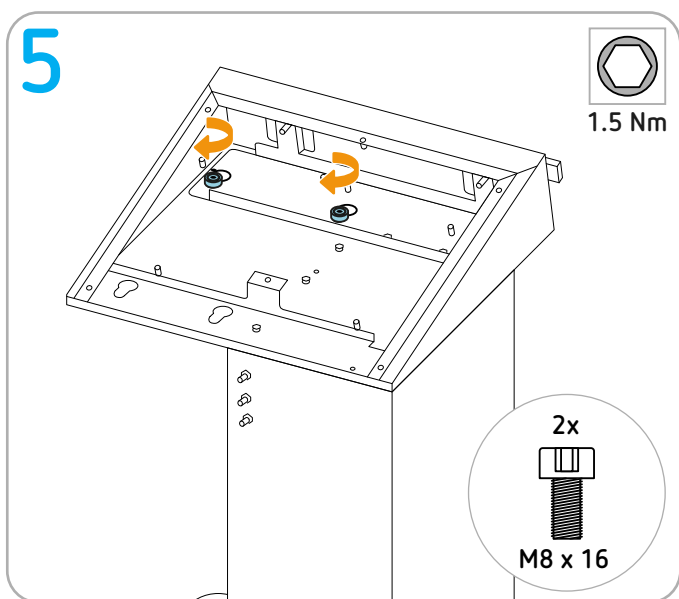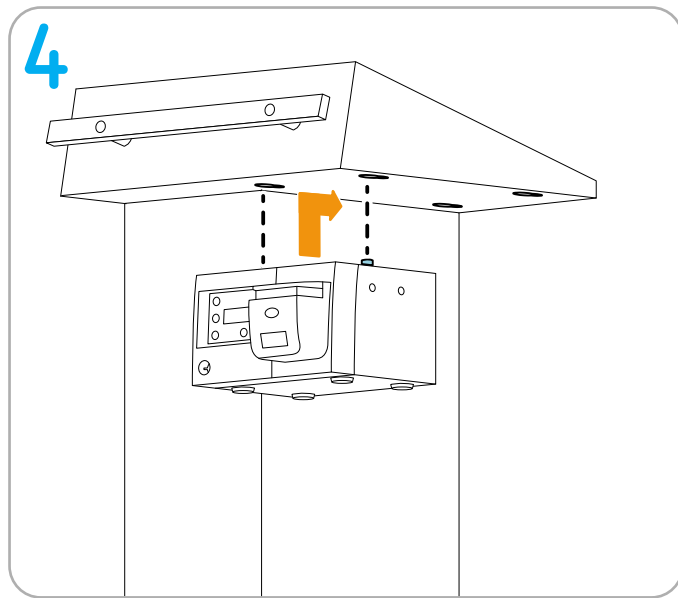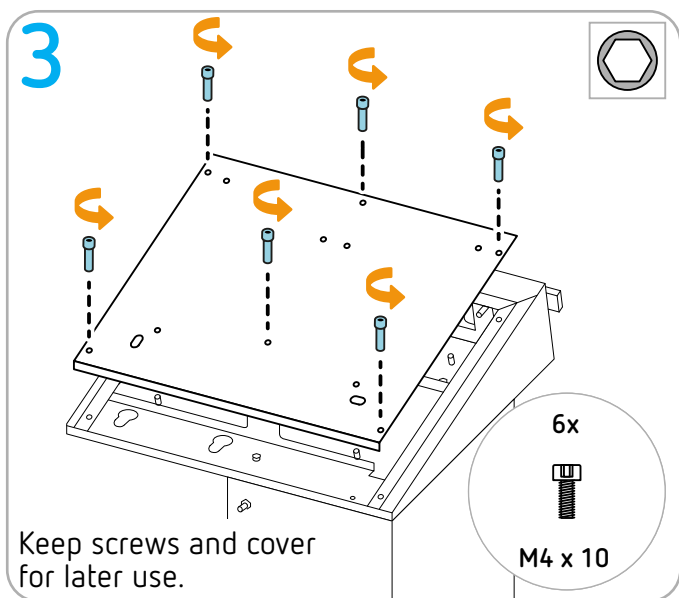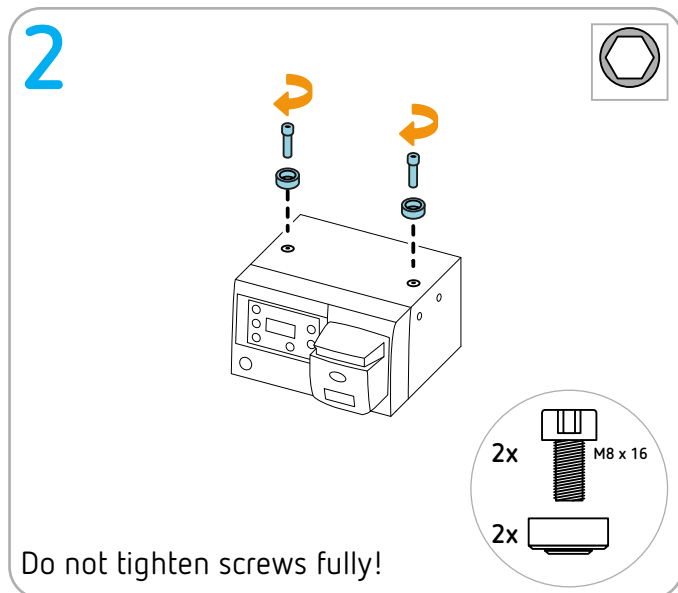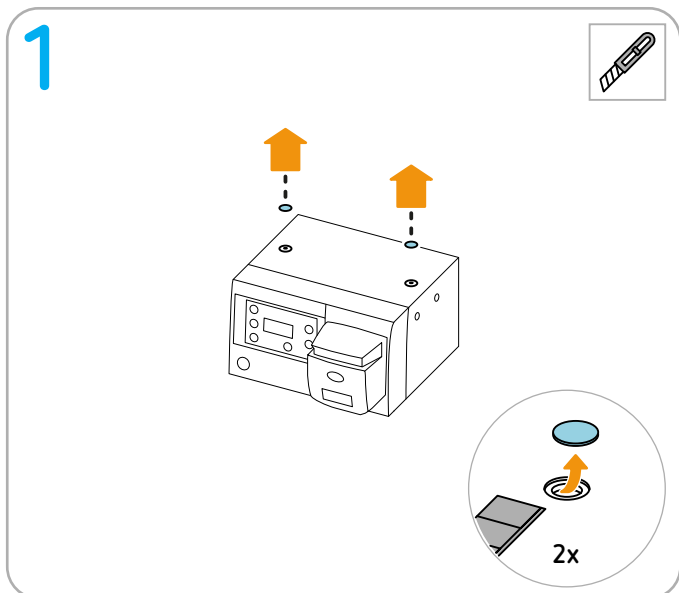

Installing VIO C series


Permitted positioning

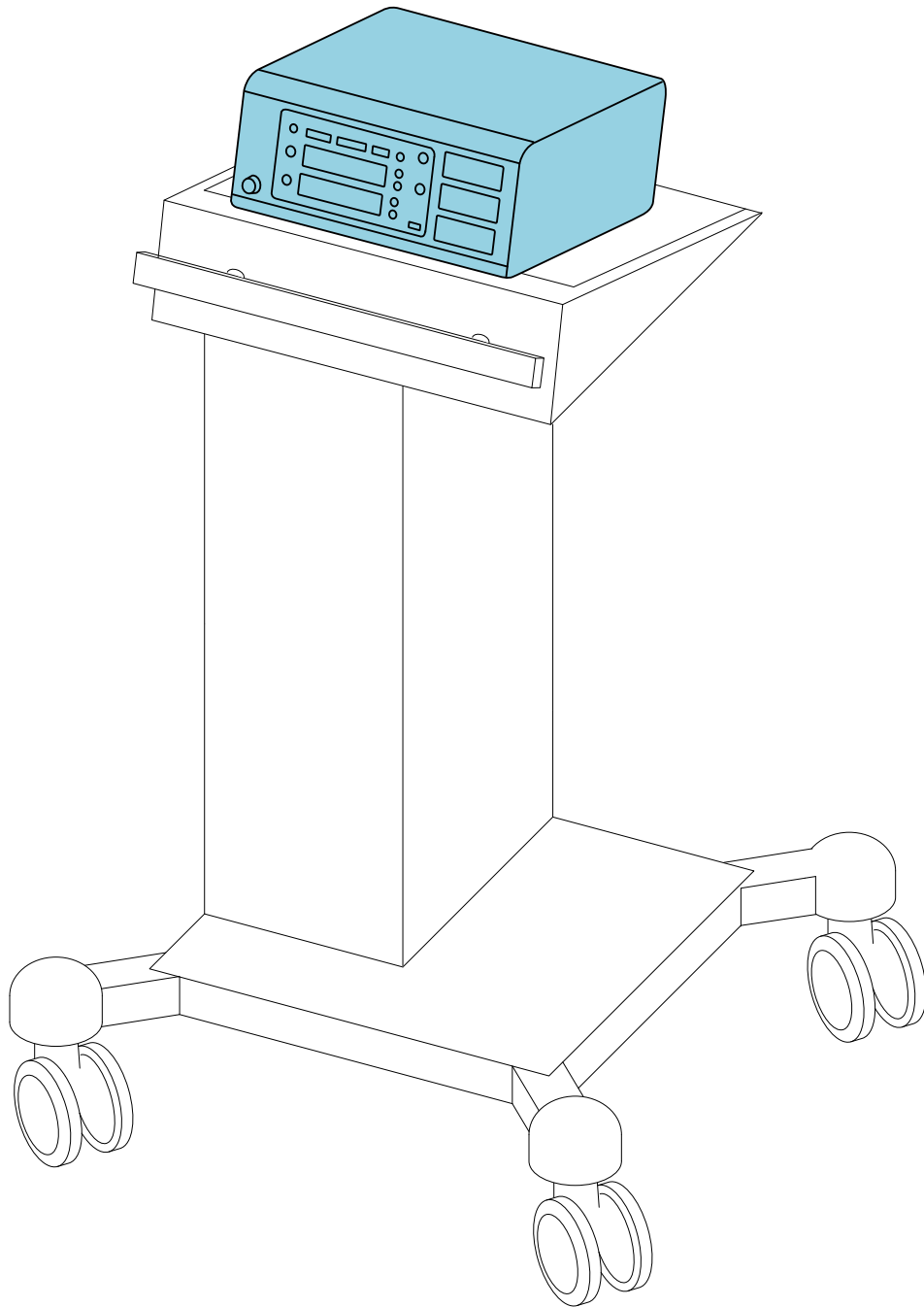Erbe fastening set

IMPORTANT! Only use original Erbe accessories.

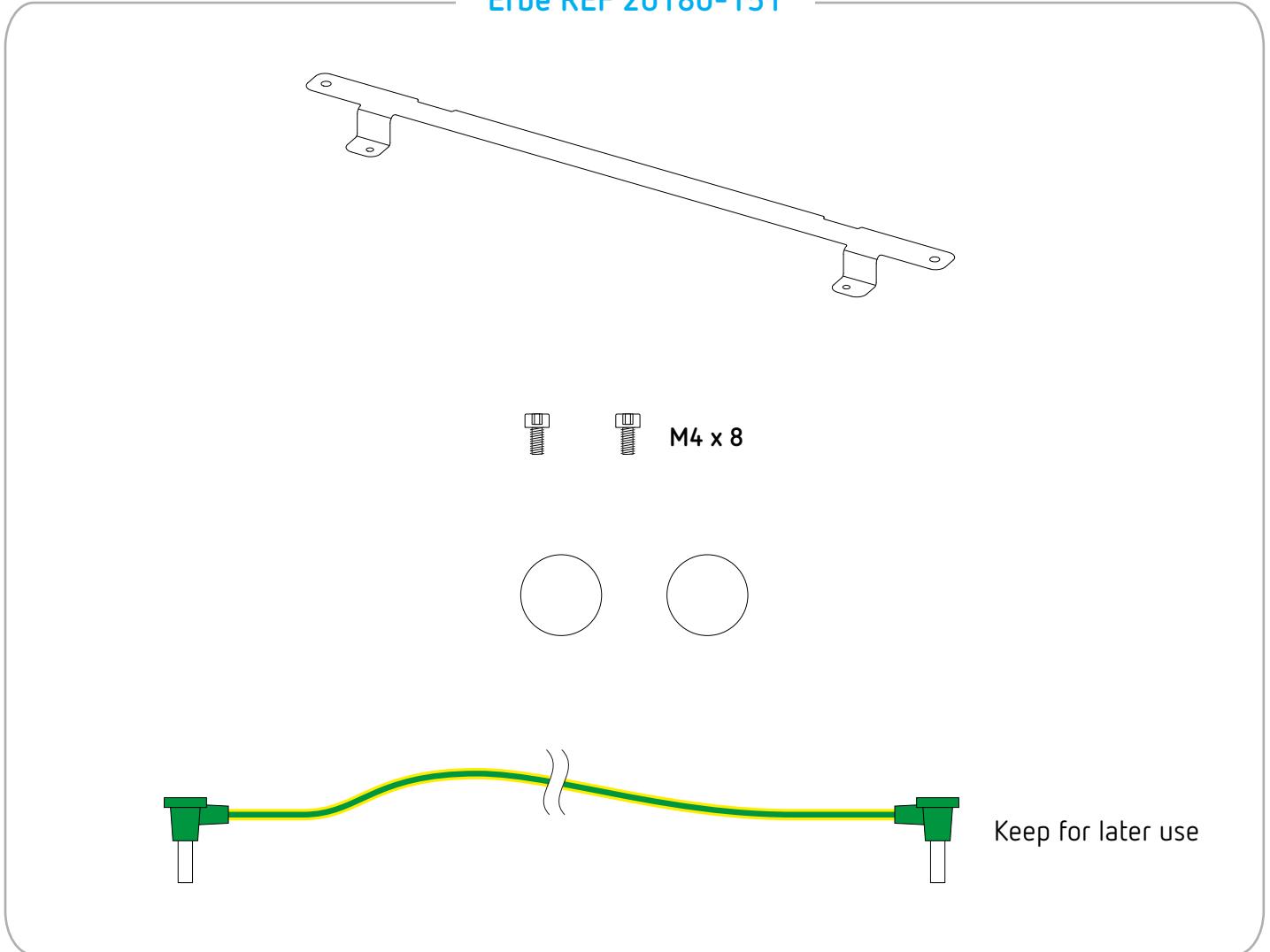Erbe REF 20180-151







Tools required

-  Torque screwdriver (1 Nm) hex socket 2.5
-  Torque screwdriver (0.5 Nm) Torx 10
-  Cutter

Preparation


IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.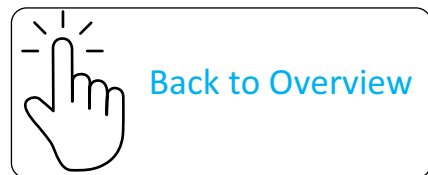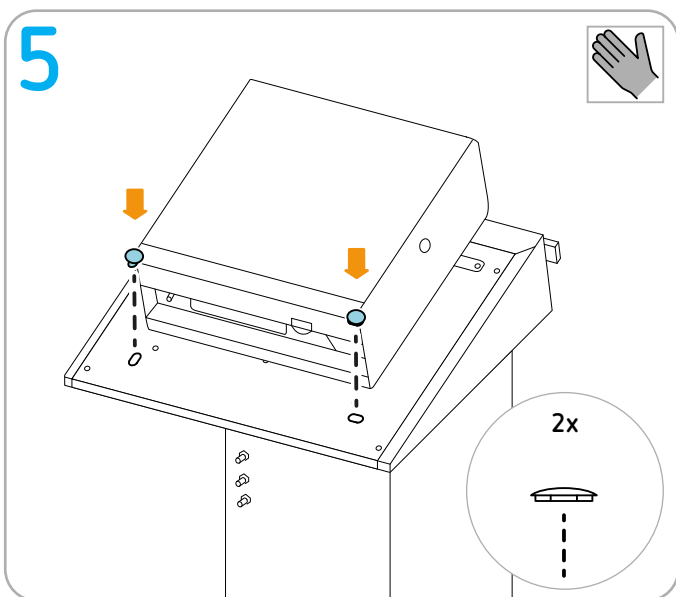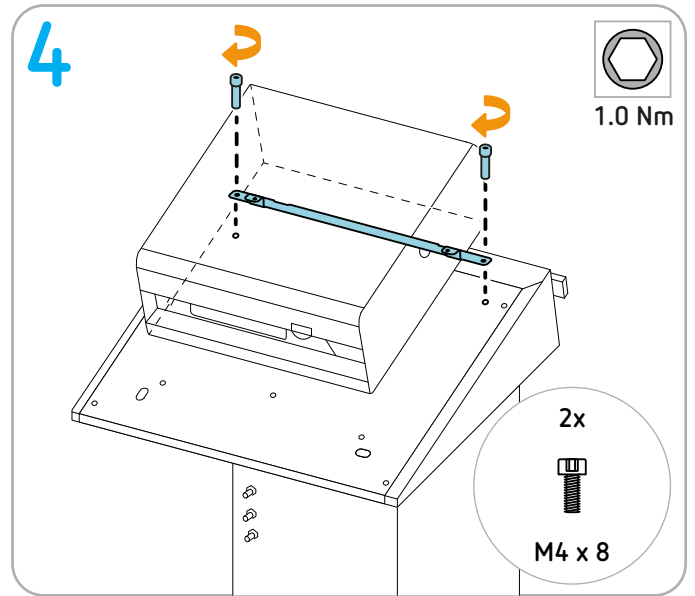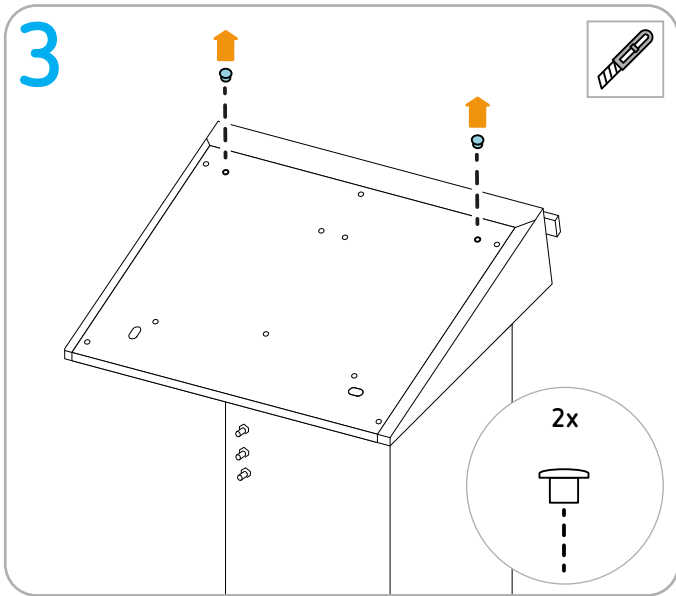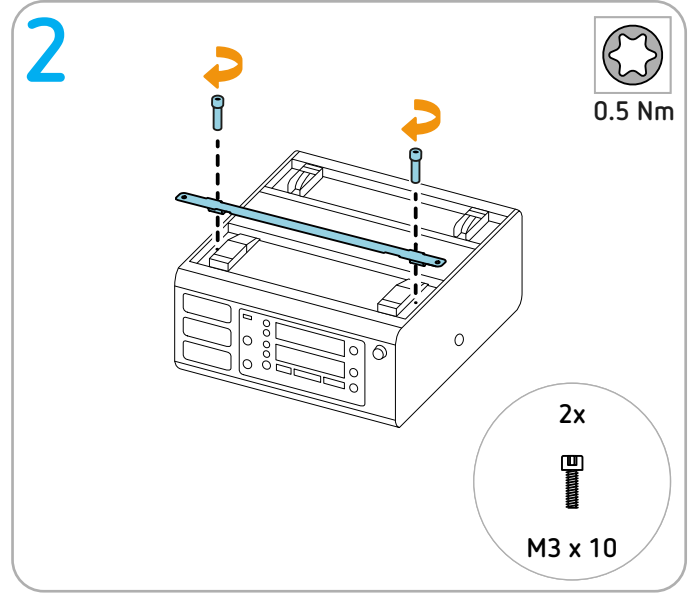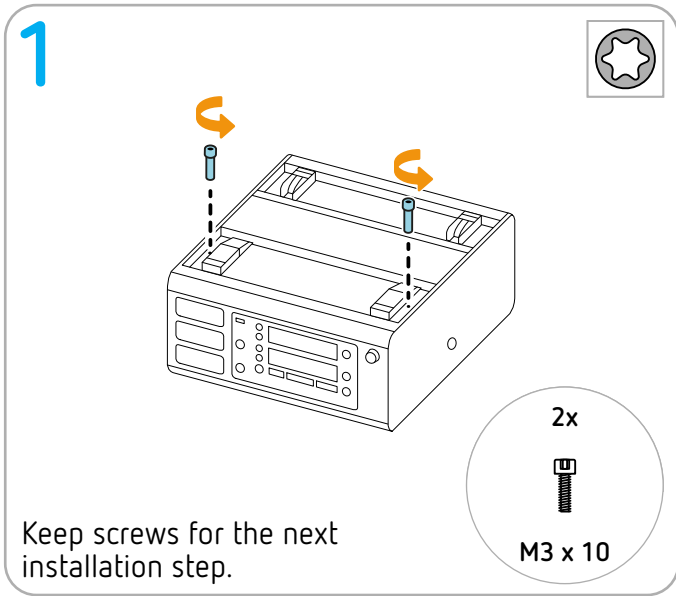


Installing VIO S, VIO D or VIO 3 series

Permitted positioning

VIO S series and
VIO D series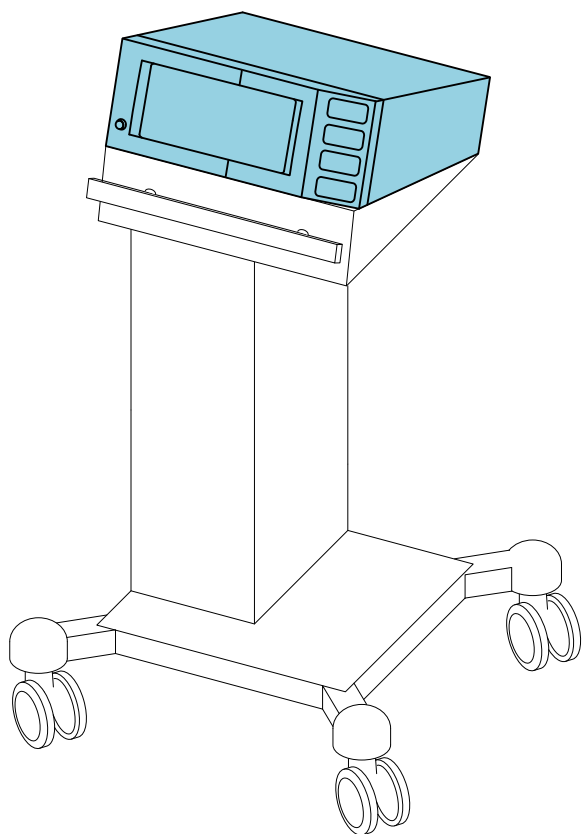

VIO 3 series

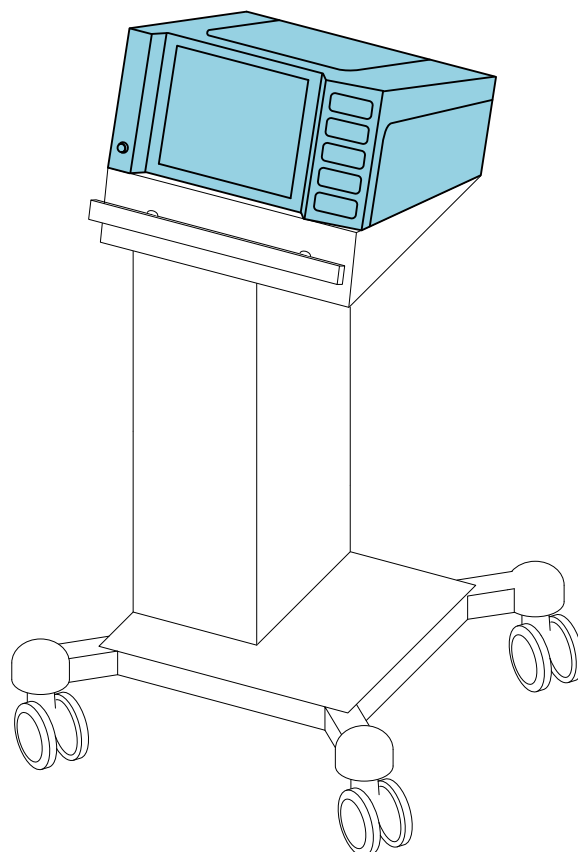Erbe fastening set



IMPORTANT! Only use original Erbe accessories.

IMPORTANT! Order the protective caps for the via of the units (see Step 1) separately as needed; they are not part of this fastening set 20180-150. Protective cap for VIO 3 via Erbe REF 40160-112 (1 item needed), protective cap for VIO S or VIO D via Erbe REF 40140-099 (2 items needed).


Erbe REF 20180-150

M4 x 8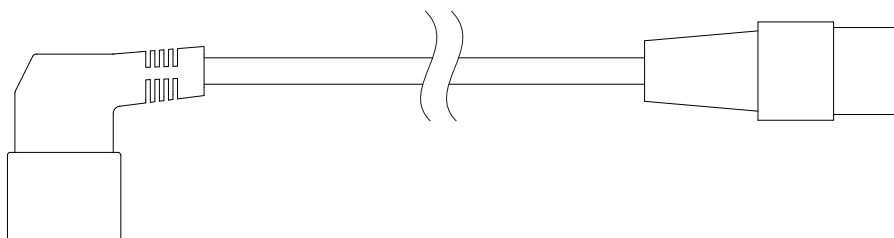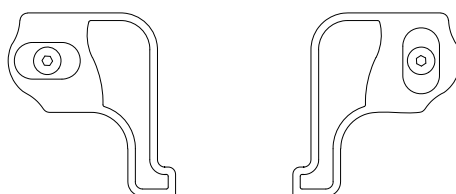

Keep for later use

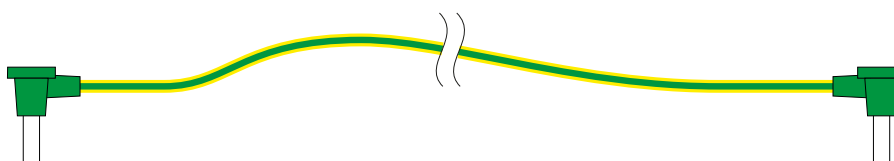Keep for later use







Tools required


Torque screwdriver (1 Nm) hex socket 2.5

Preparation


IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.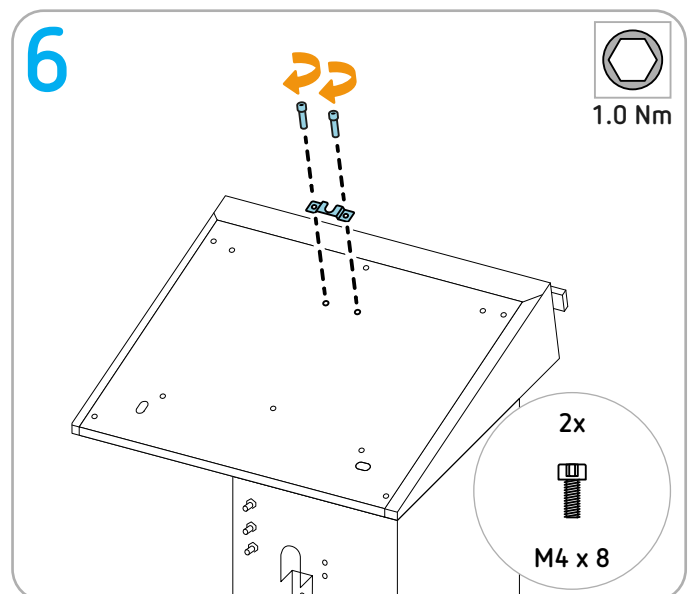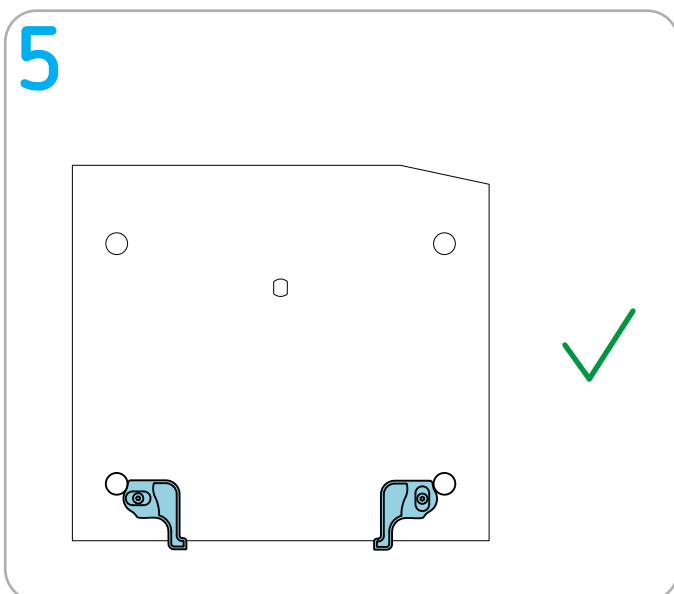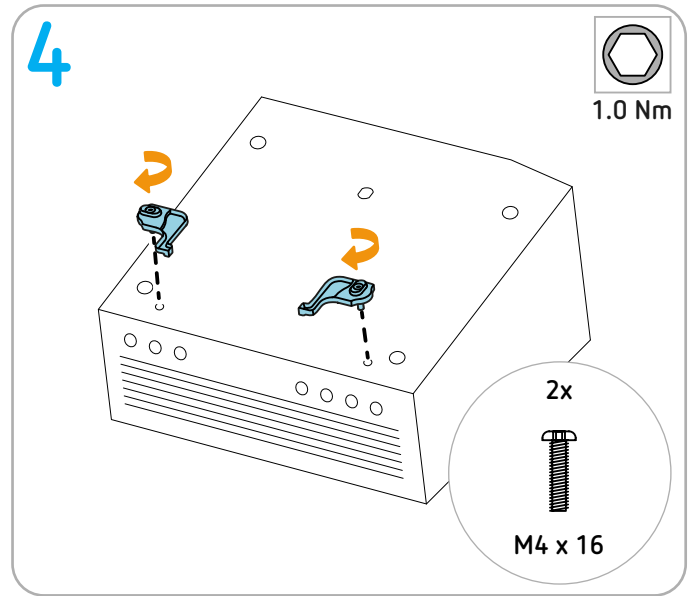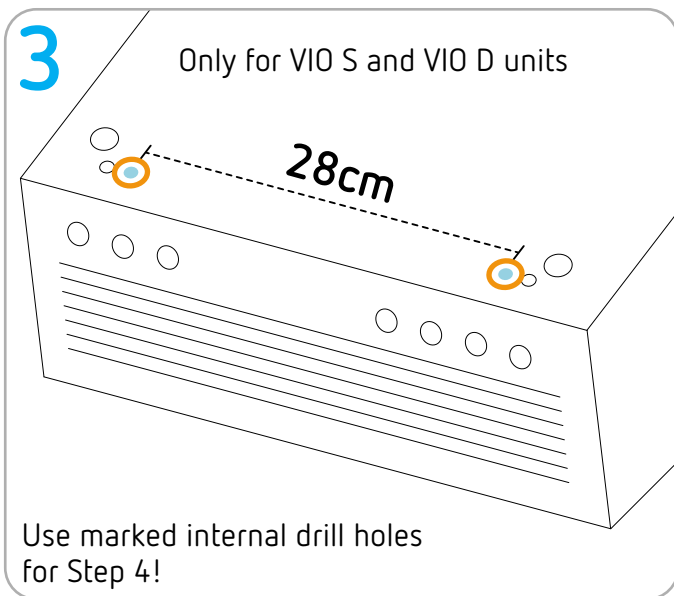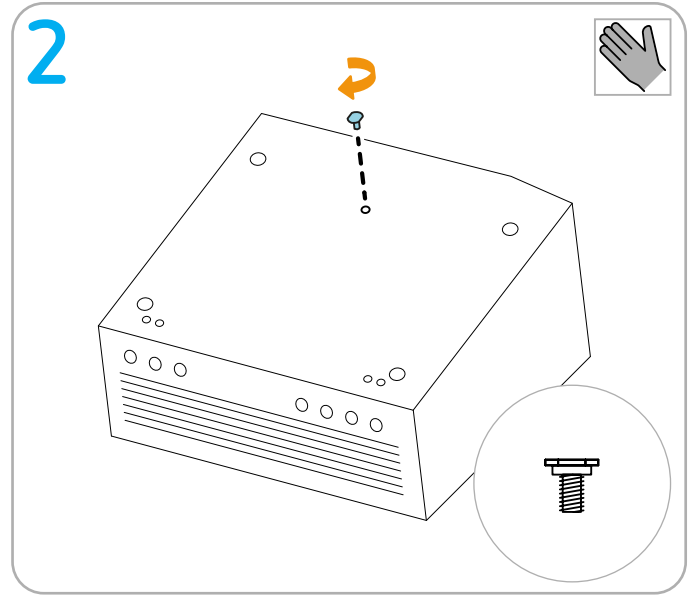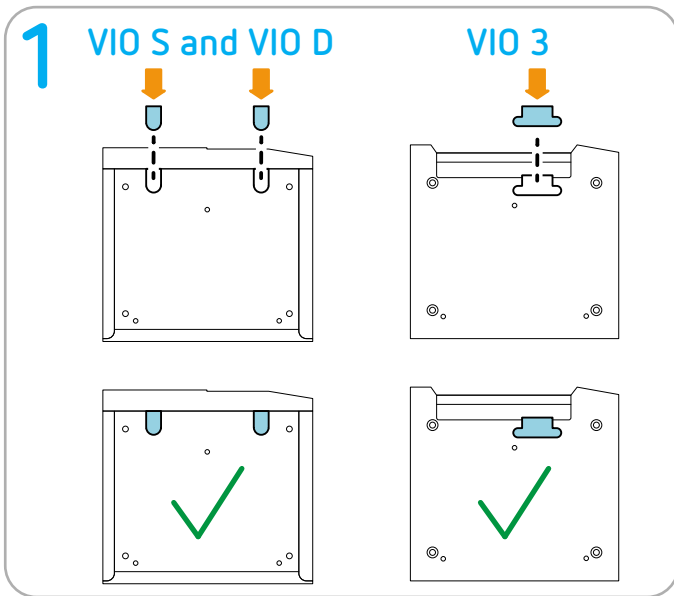


7 Attachment points on the system carrier

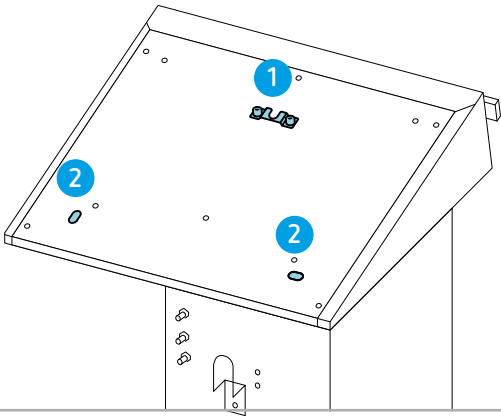
8 Attachment points on the unit


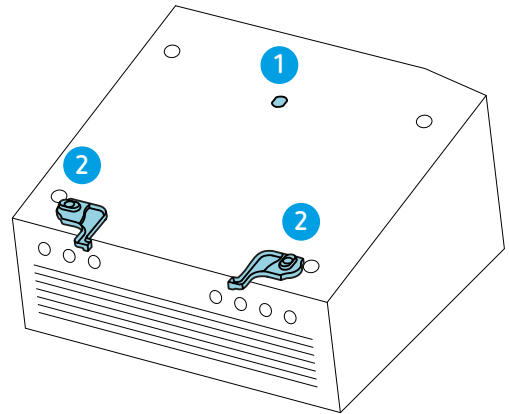
9

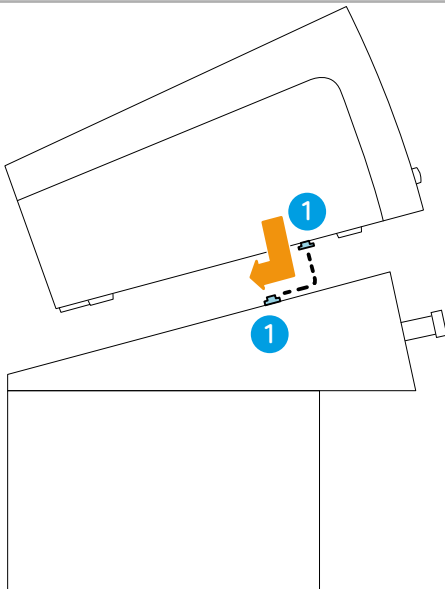10

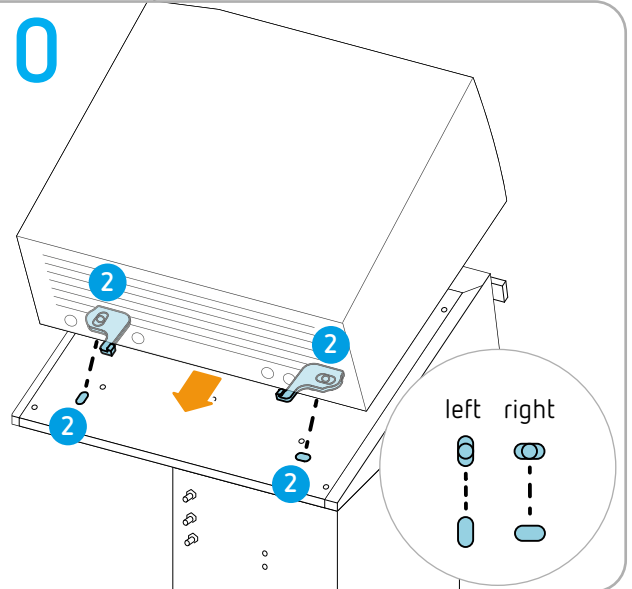
11

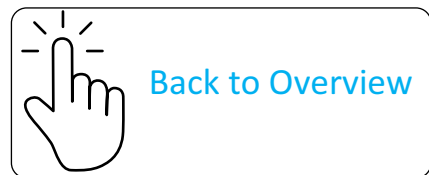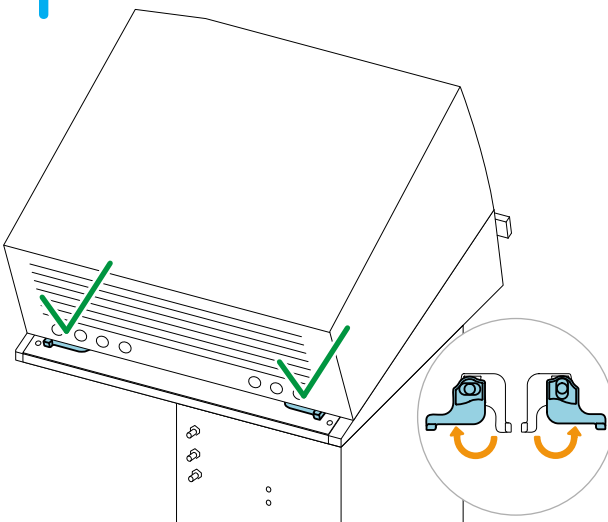
Only compact plus: Installing APC 2 and VIO S or VIO D series

Permitted positioning

VIO S series or VIO D series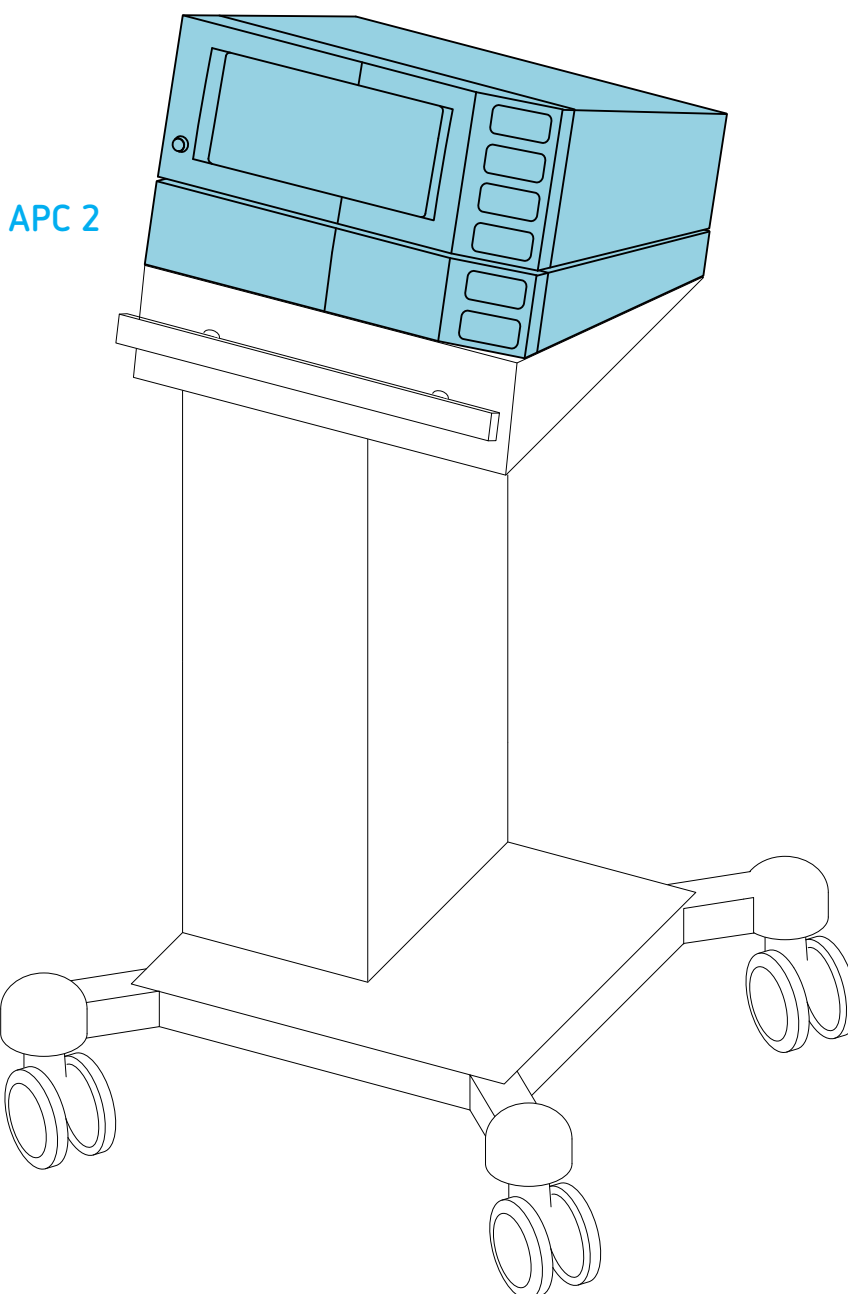

Tools required

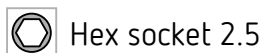Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.



Erbe fastening sets

IMPORTANT! Only use original Erbe accessories.


Fastening set for APC on system carrier Erbe REF 20180-150

M4 x 8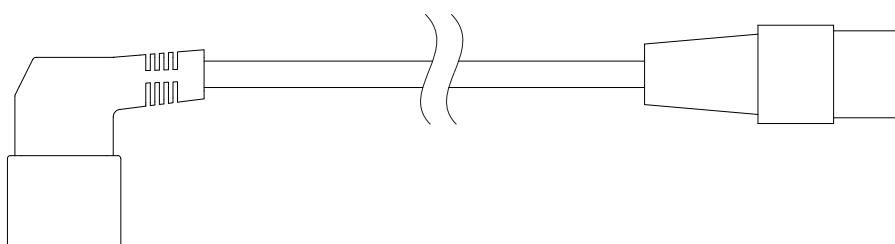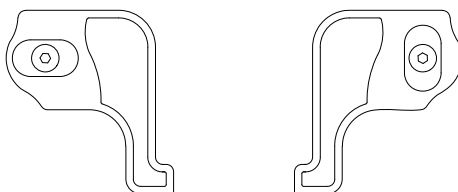


Keep for later use






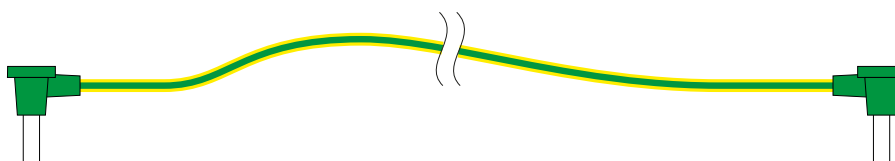
Keep for later use


**Fastening set for VIO on APC
Erbe REF 20180-134**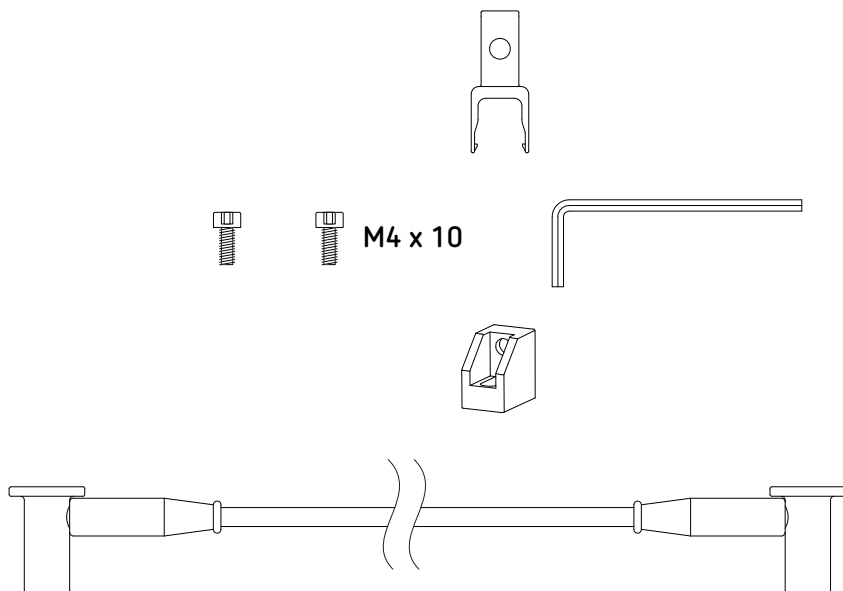


Keep for later use

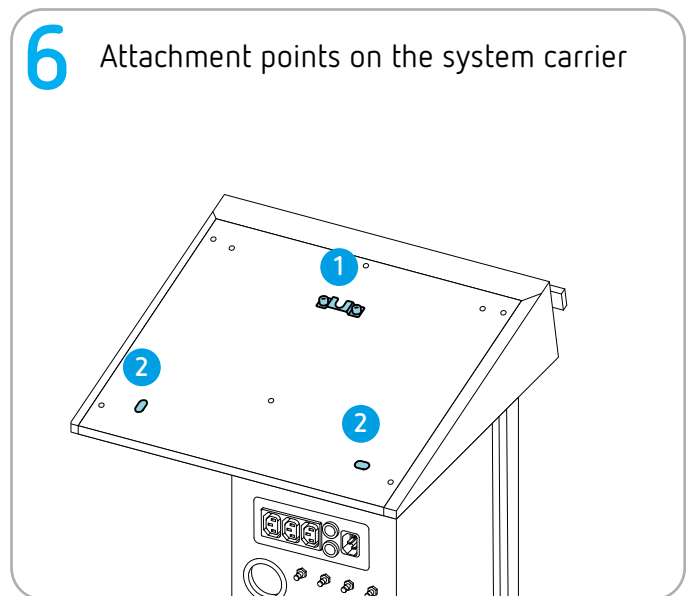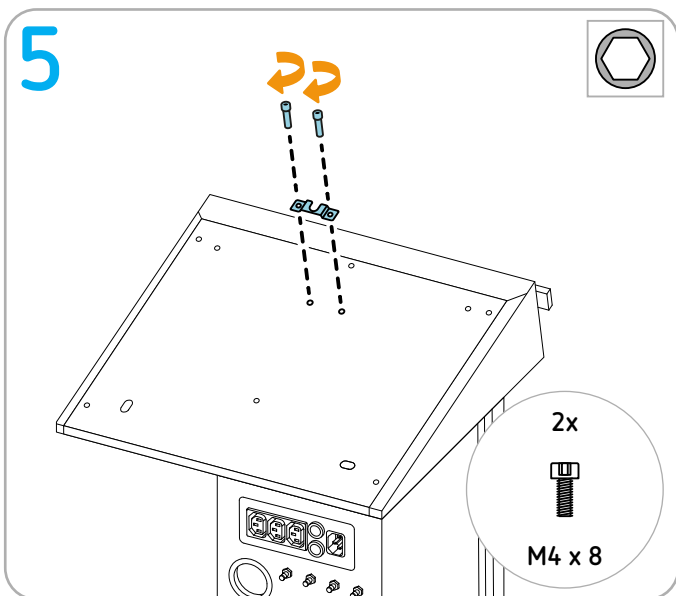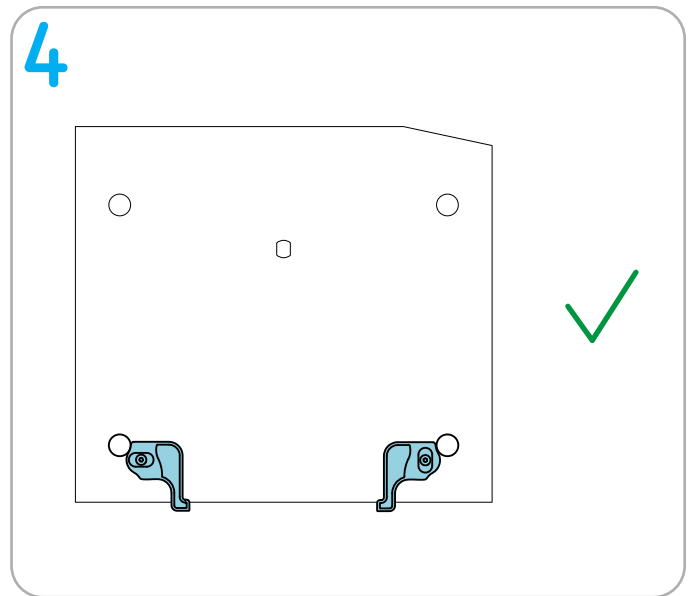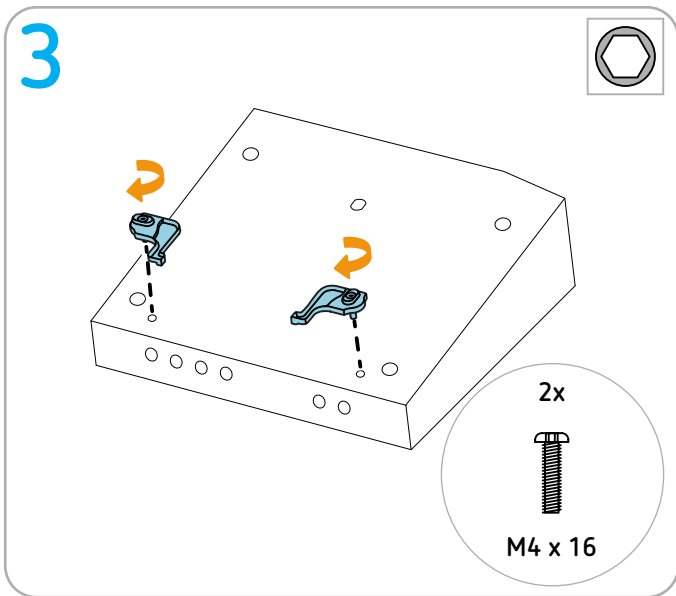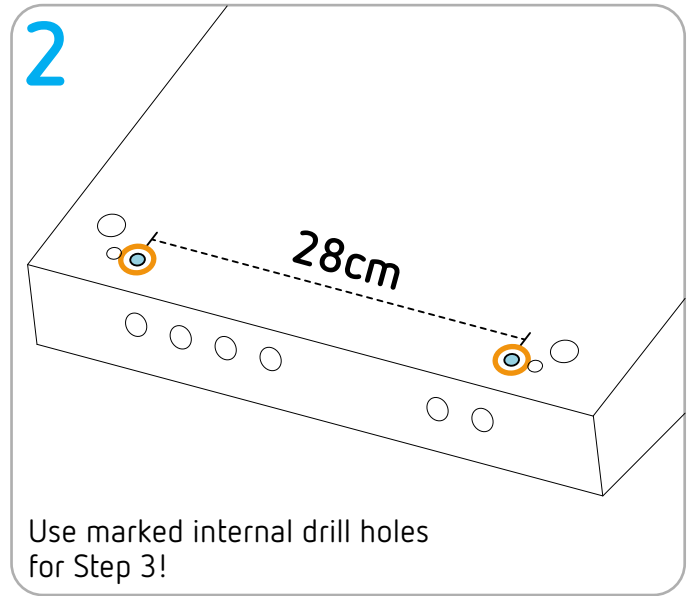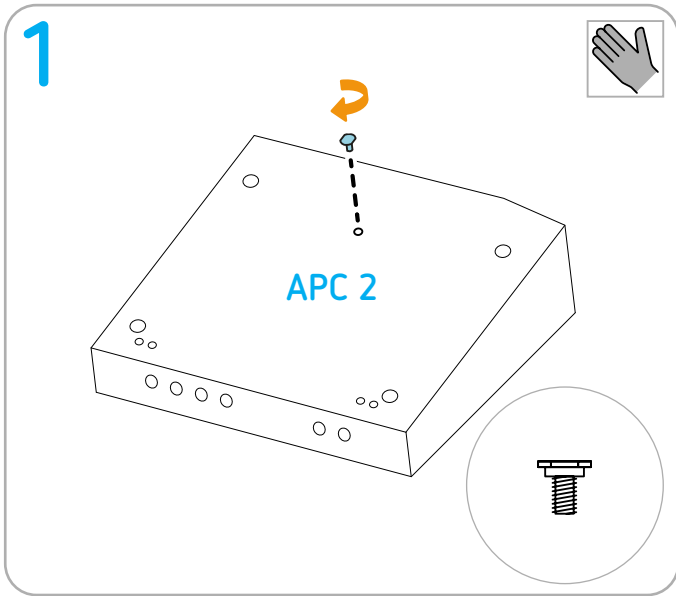
7 Attachment points on the unit

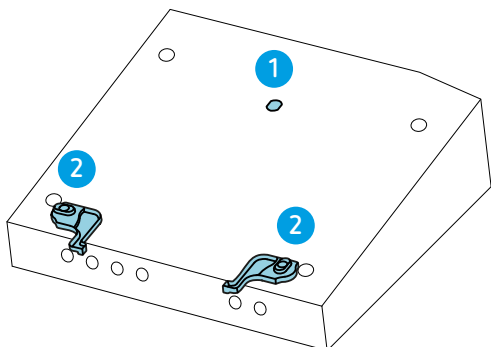
8




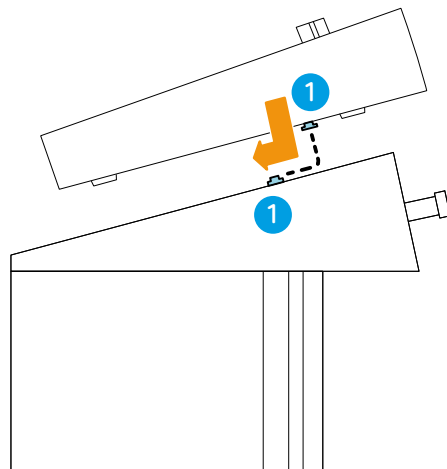
9

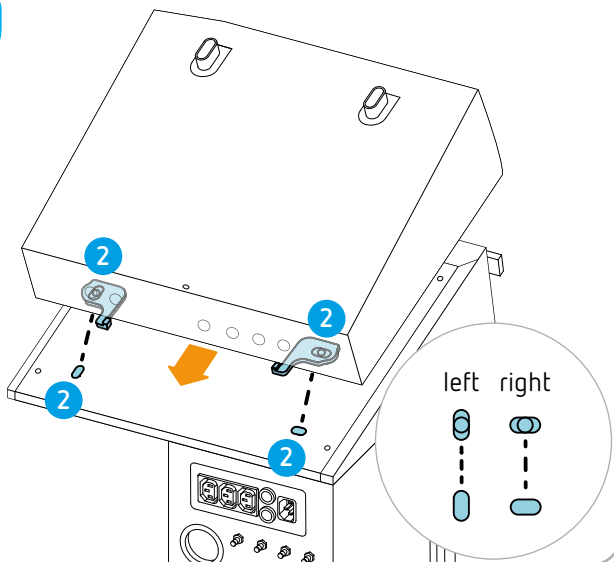10

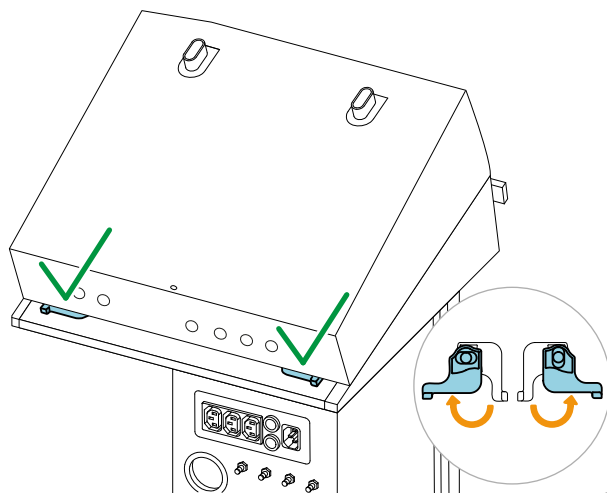
Continue to next page

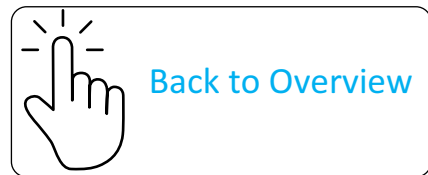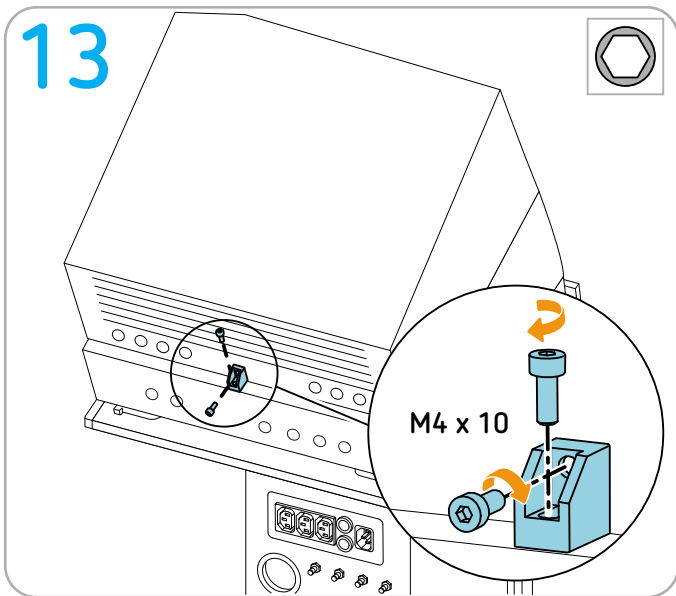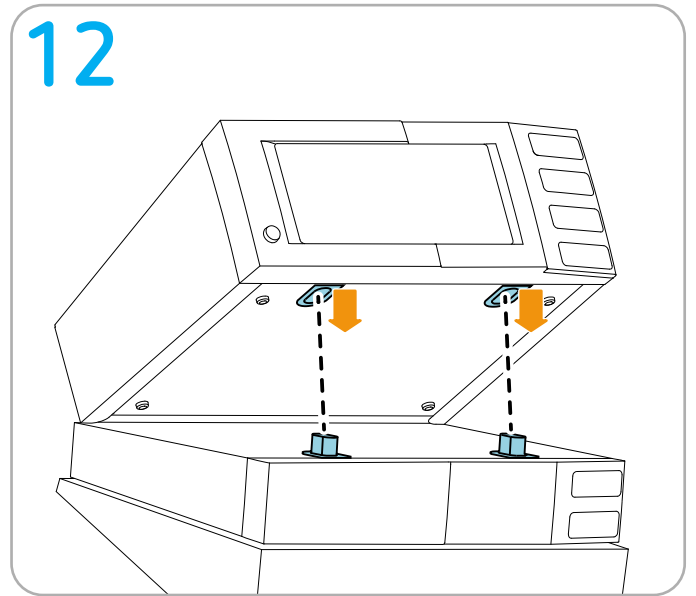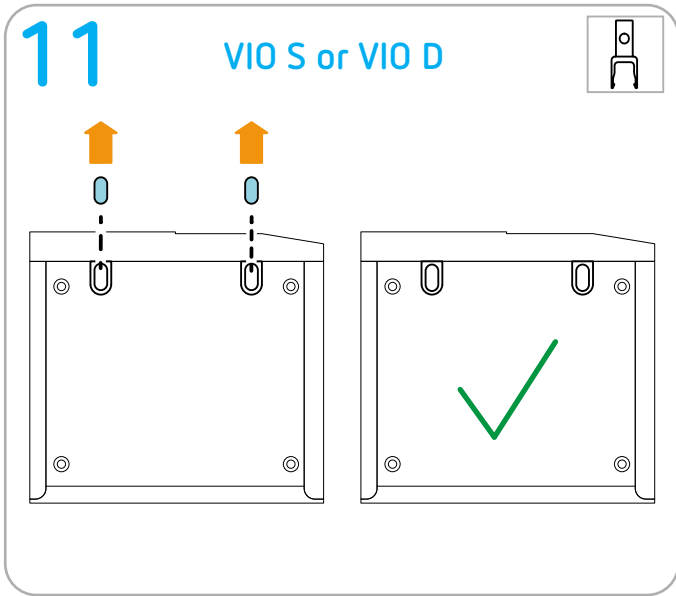
Only compact plus: Installing APC 3 and VIO 3 series

Permitted positioning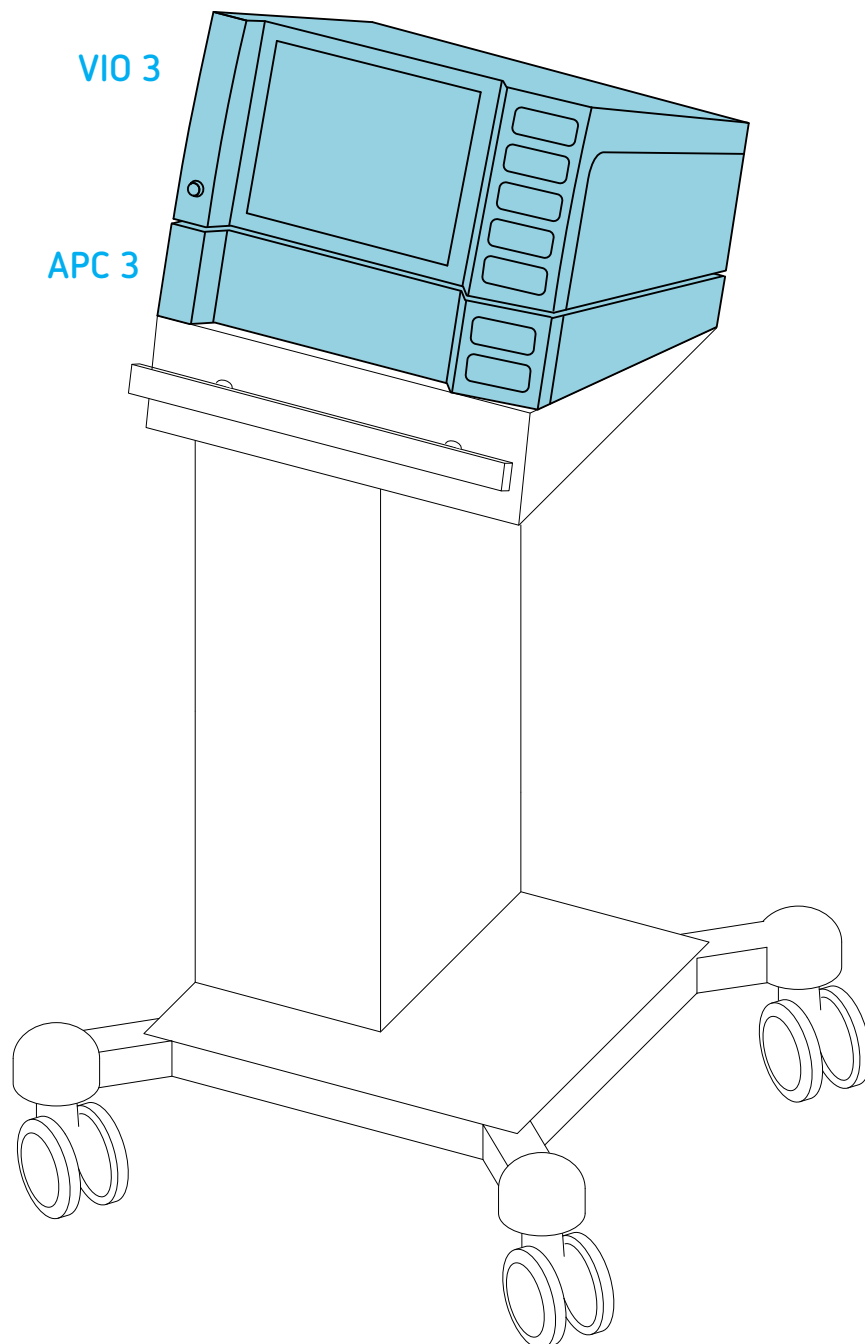

Tools required

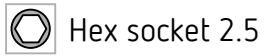Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.




Erbe fastening sets

IMPORTANT! Only use original Erbe accessories.


Fastening set for APC on system carrier Erbe REF 20180-150

M4 x 8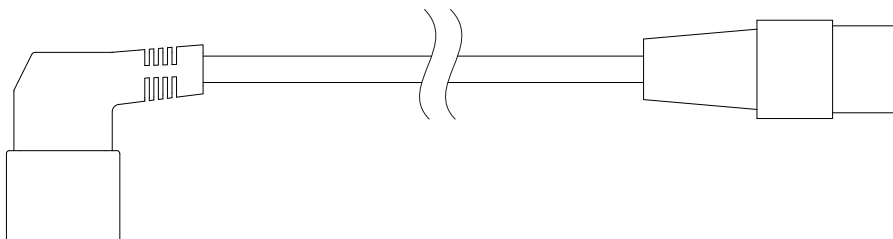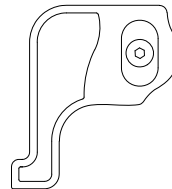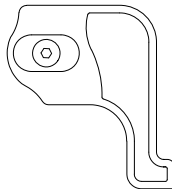


Keep for later use

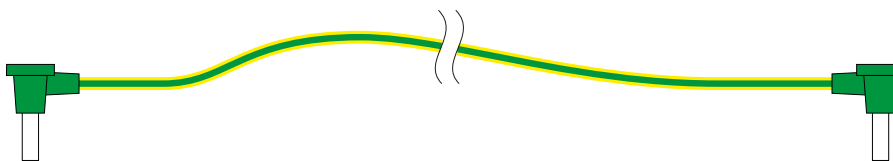Keep for later use


Fastening set for VIO on APC: Erbe REF 30135-030
Bracket: Erbe REF 40160-065

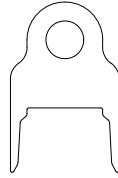
M4 x 8







M4 x 16

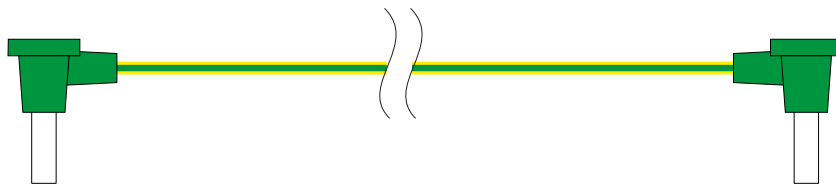


Keep for later use





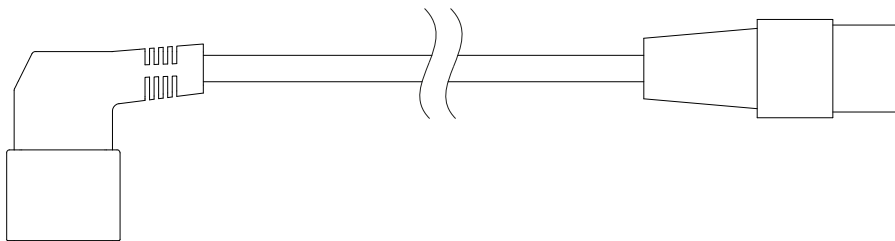
Keep for later use

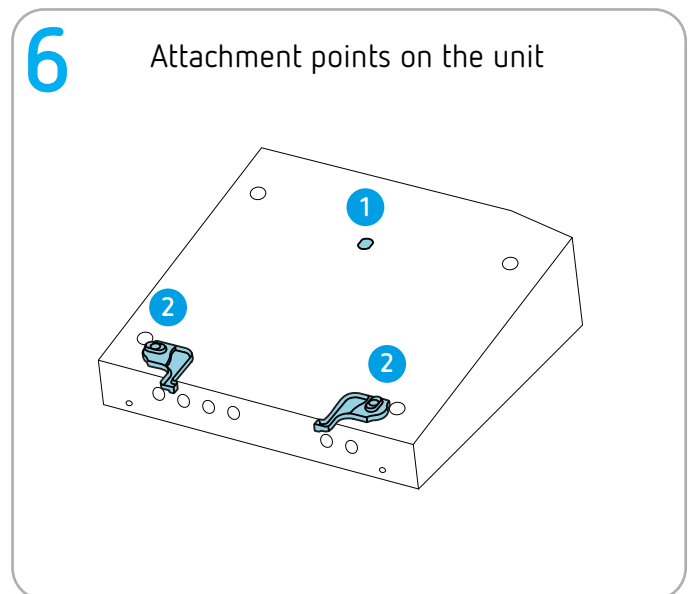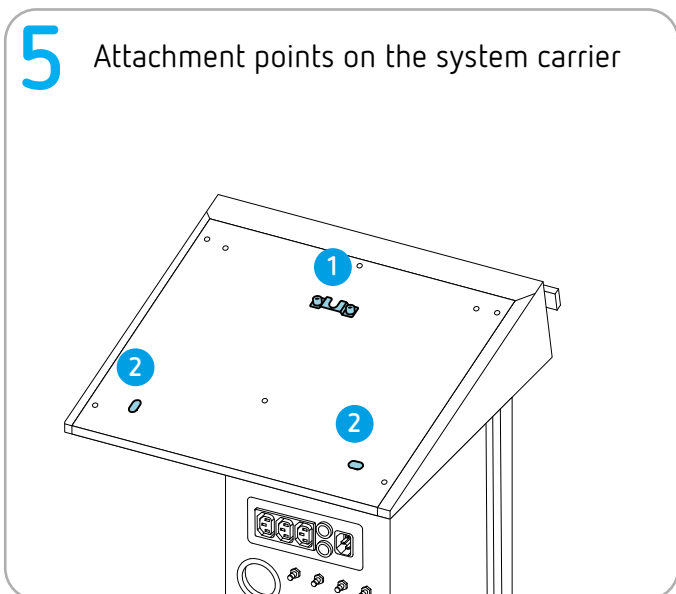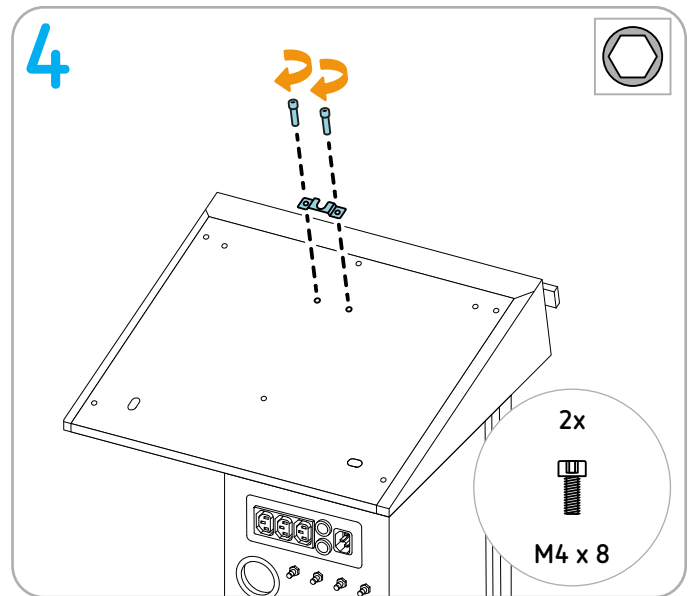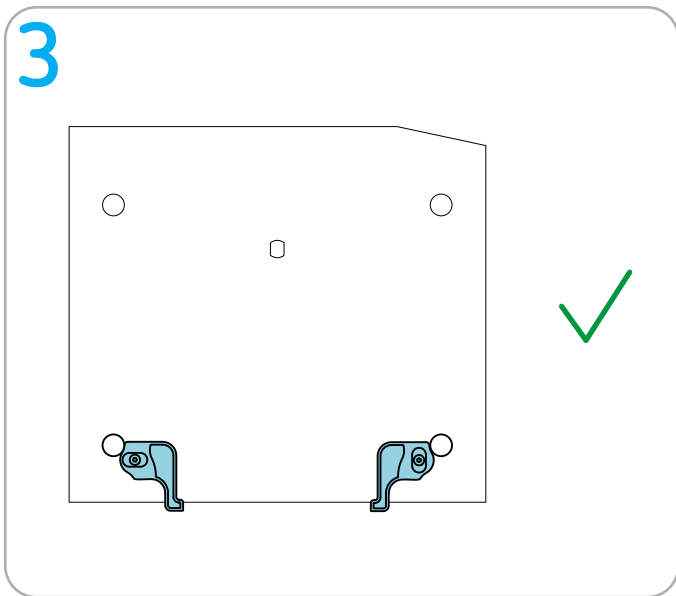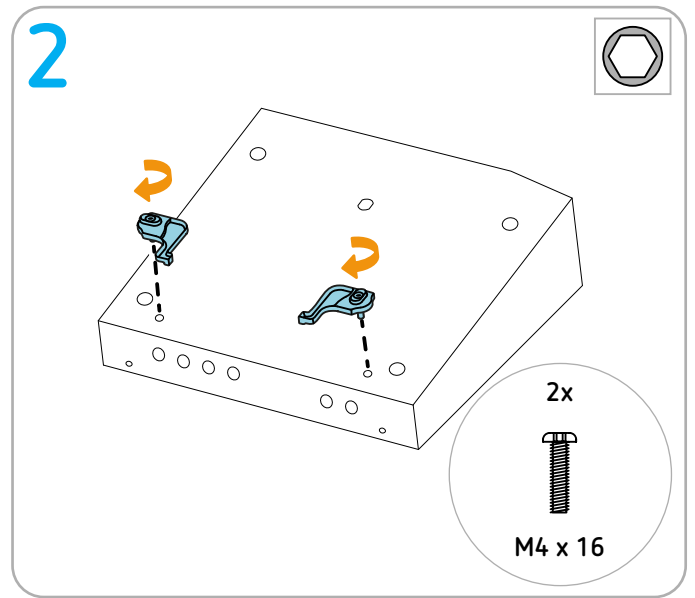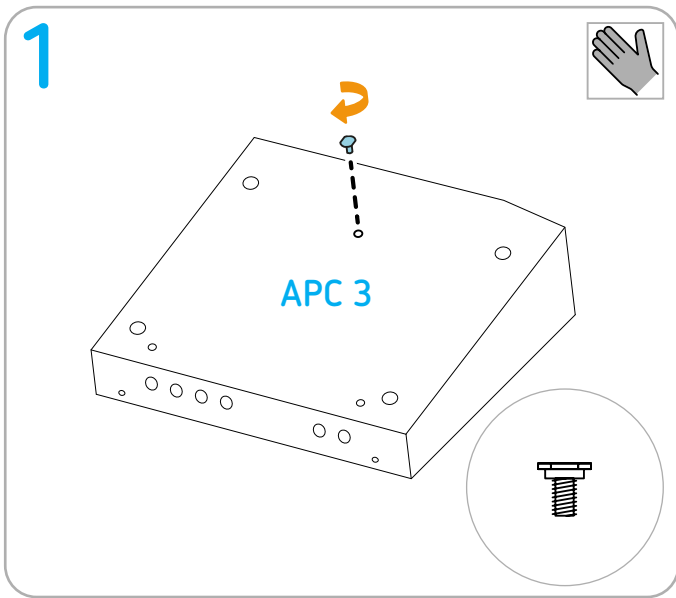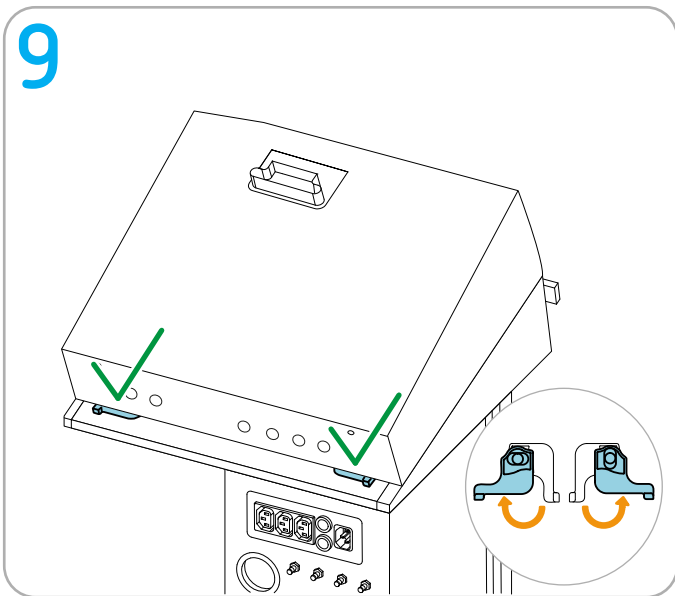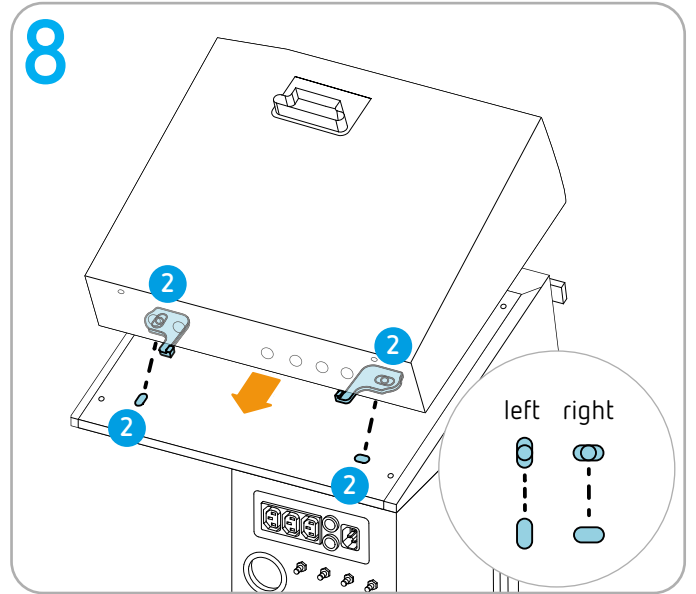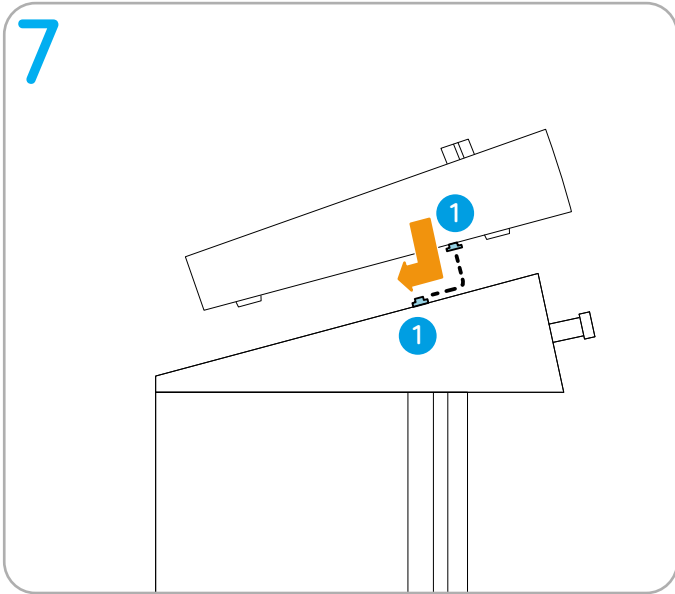
Continue to next page

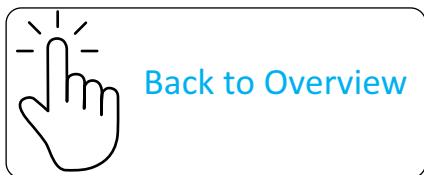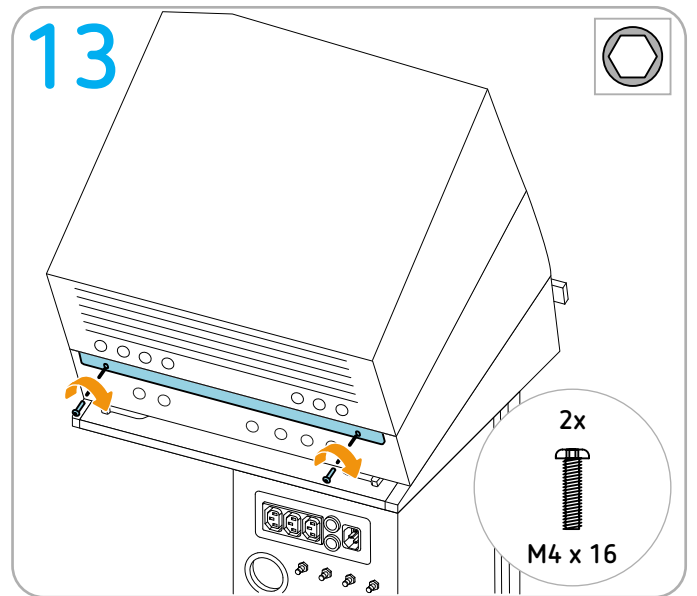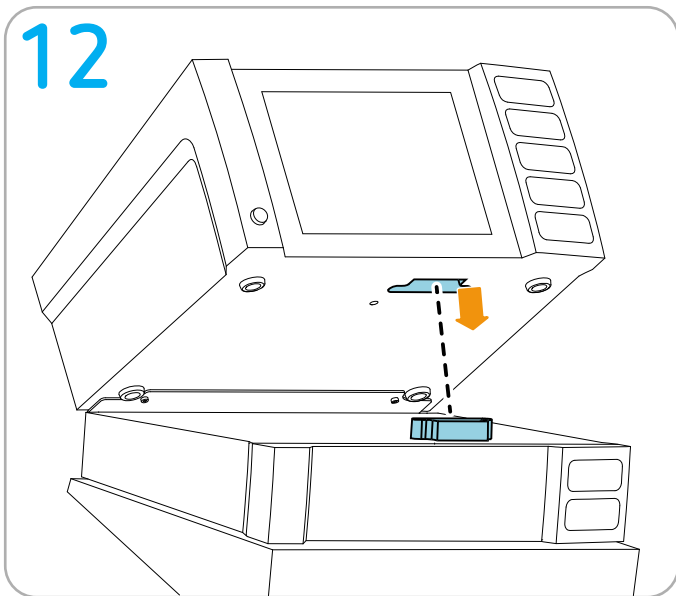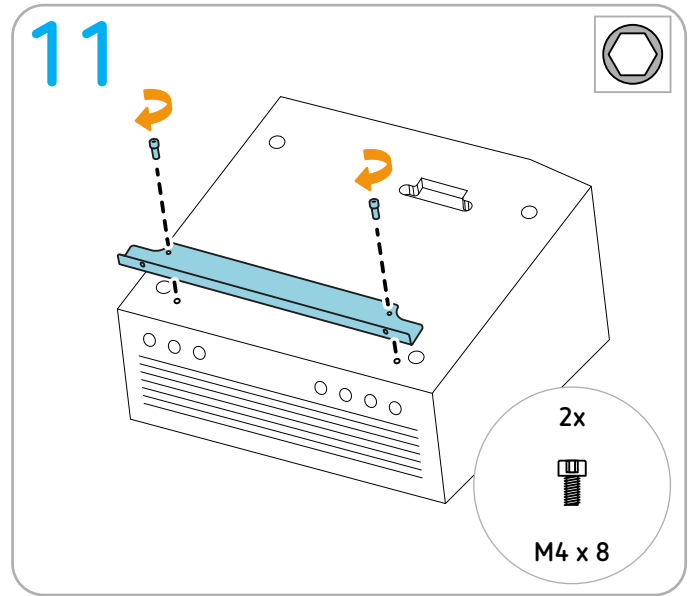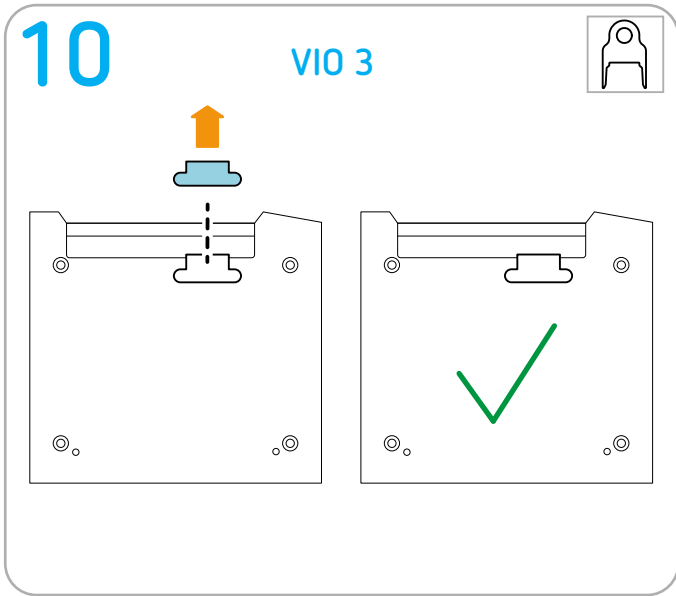Only compact plus: Stowing argon gas bottle

Permitted positioning

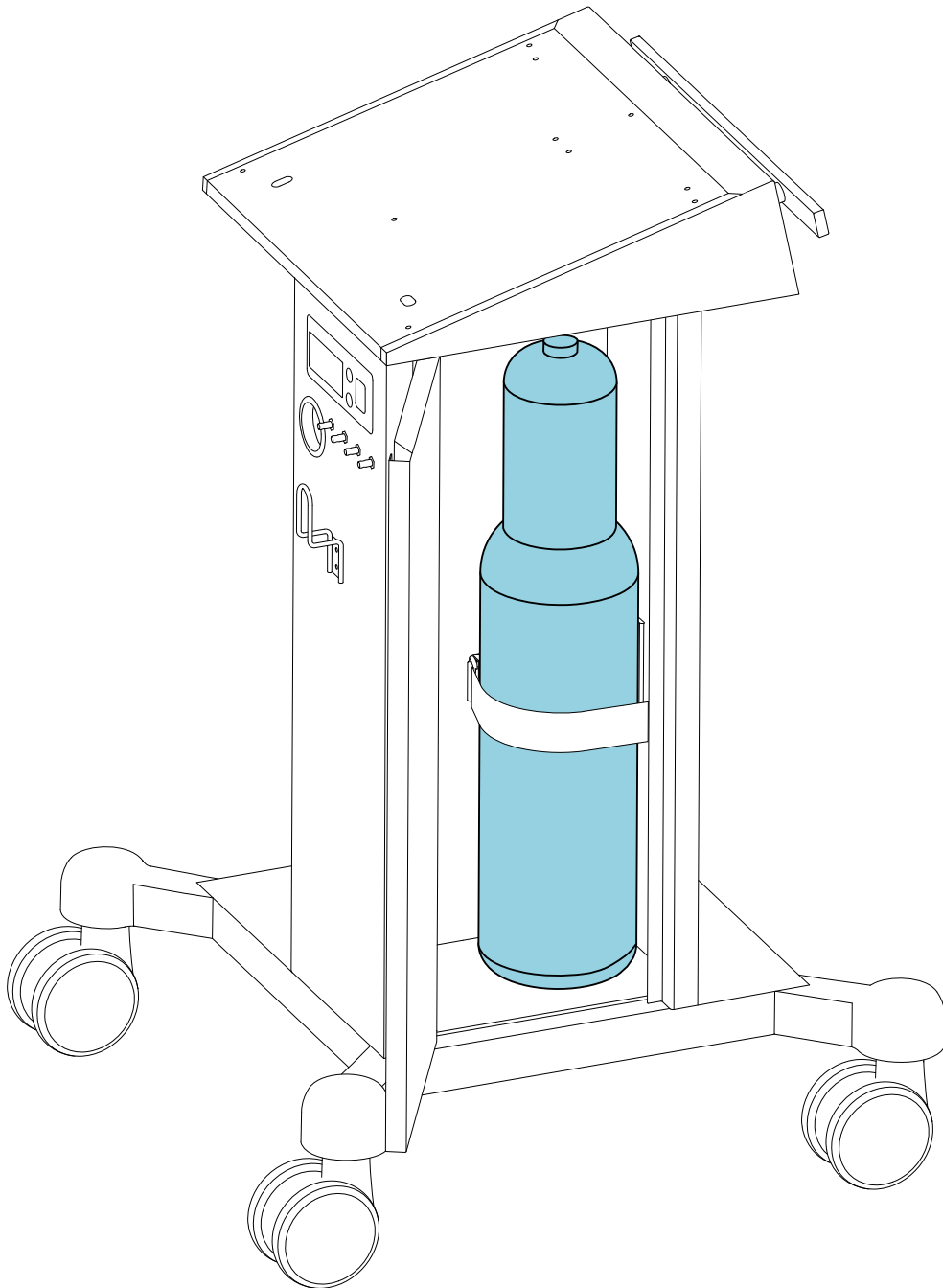


Tools required

No tool needed.

Preparation


IMPORTANT! Remove the power cord from the system carrier.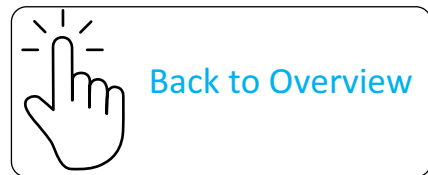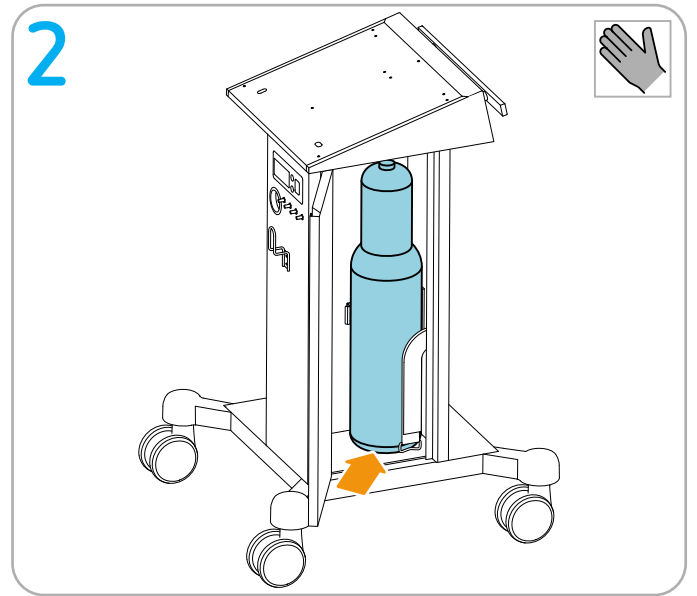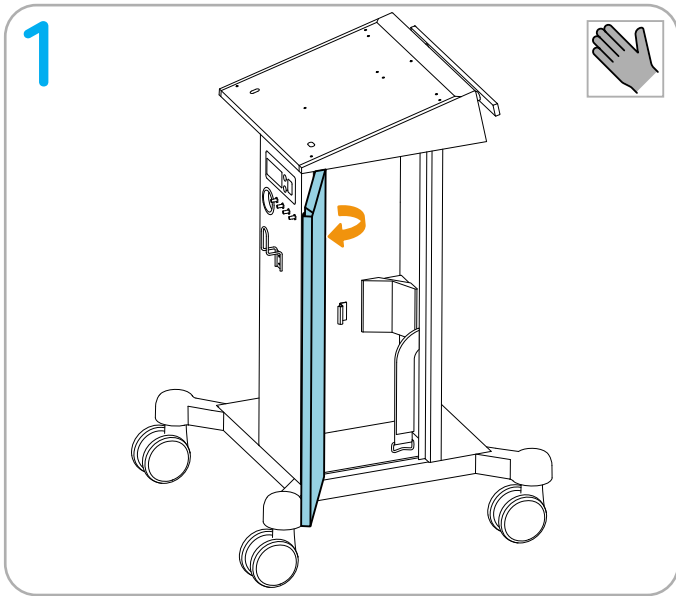

compact: Making connections


Potential equalization and mains connection

IMPORTANT! Only use original Erbe cables.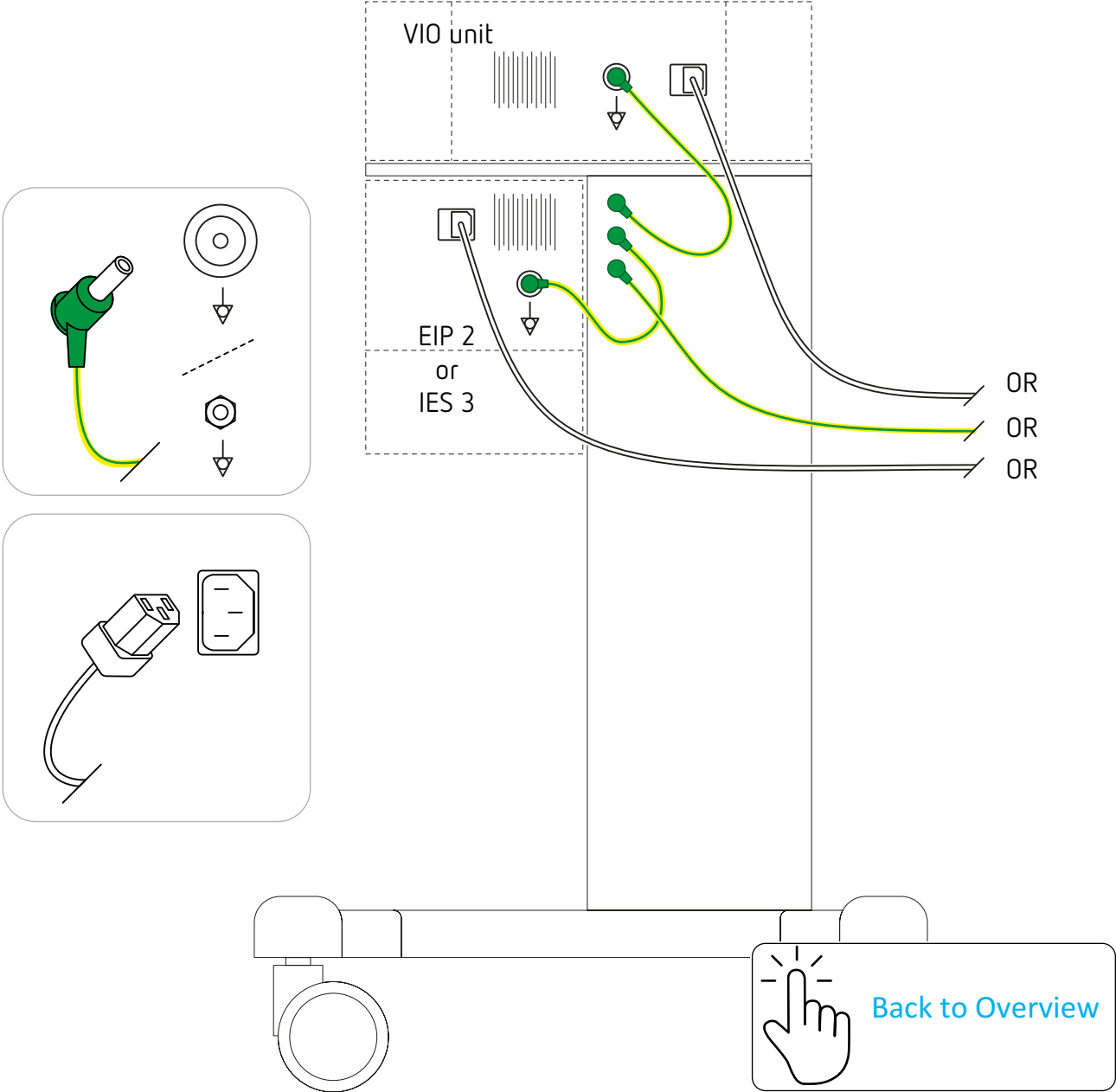

ECB connection (optional, for communication between the units)


IMPORTANT! Only use original Erbe cables.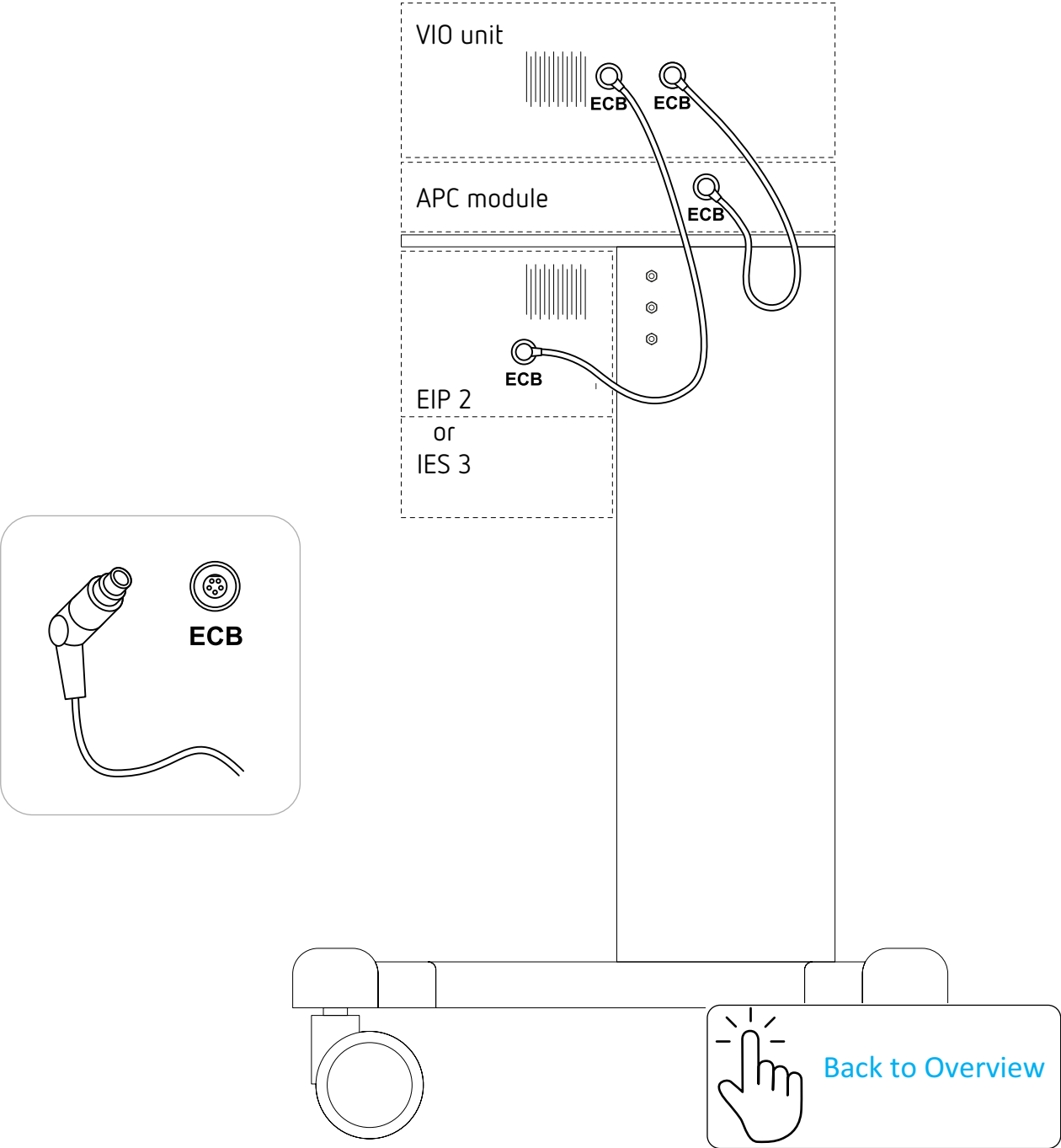

compact plus: Making connections


Potential equalization and mains connection

IMPORTANT! Only use original Erbe cables.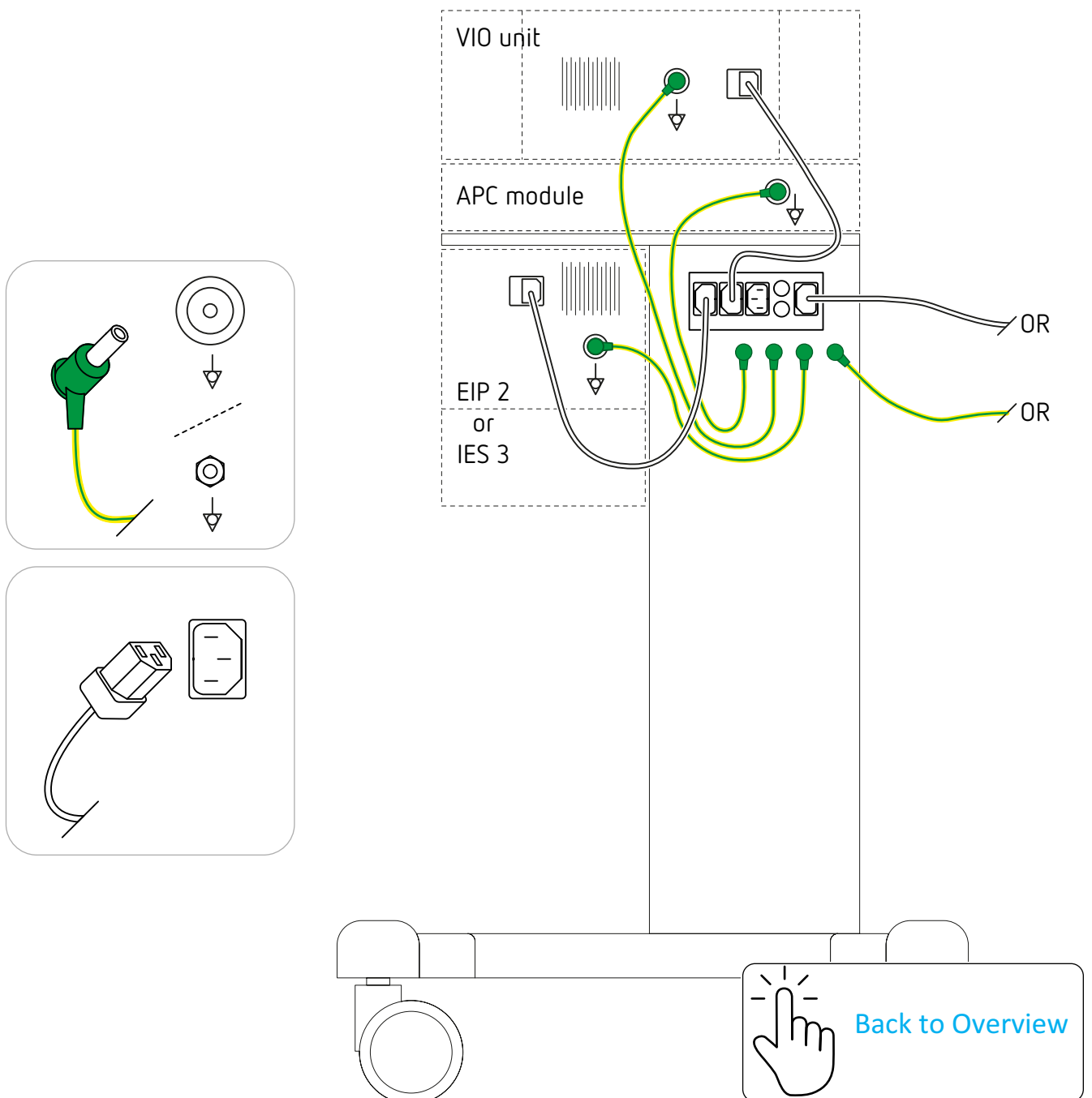


ECB connection (optional, for communication between the units)

IMPORTANT! Only use original Erbe cables.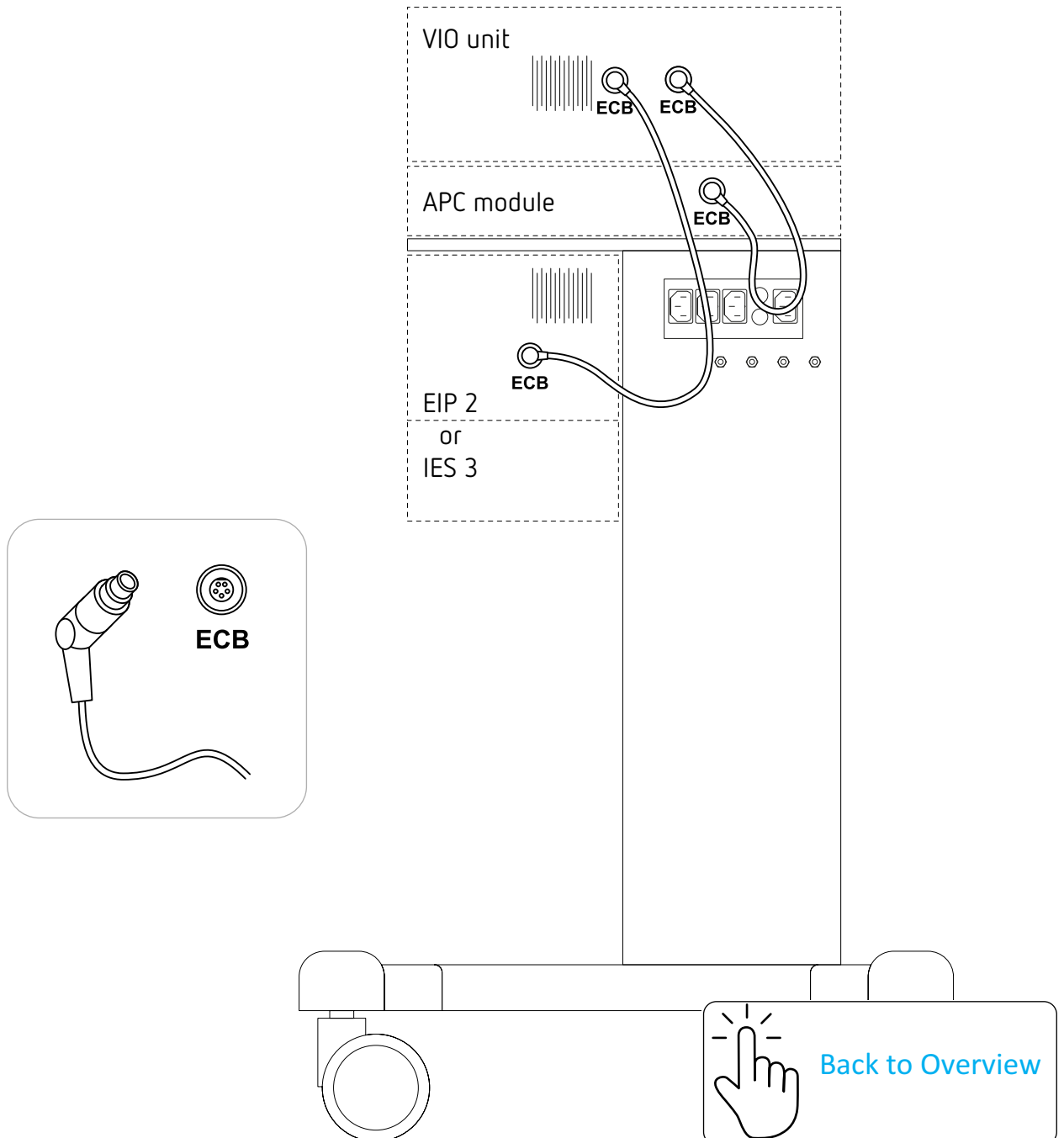

APC connection

Please read the User Manual for the APC 2 or the User Manual for the APC 3.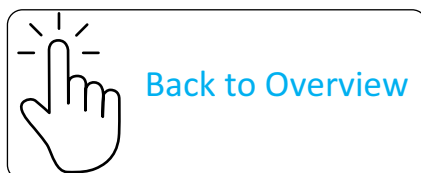

 **Manufacturer**
Erbe Elektromedizin GmbH
Waldhörnlestr. 17
72072 Tübingen, Germany
erbe-med.com

Service Hotline
Tel +49 7071 755 437
Fax +49 7071 755 189
techservice@erbe-med.com